

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006676

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3003</u> Lot No. <u>201610A</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3063</u> Lot No. <u>16102</u> Rep AC <u>0.400</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR5135</u> Lot No. <u>IC54</u> Solution <u>0.10AC + 0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>17316080A2</u>	Cyl No. <u>26</u>	Exp. Date <u>8/5/2018</u>
Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

J. Tank pressure, Level 3 Func D Sub G P: 218 psi Reg 250 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4/12/2017
Date

12 Apr 17
Date

Charles E. Egan
Field Inspector's Signature

Dob Sherover
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:16

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 26
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 08/22/2016
Slope 710
Intercept -713681

IR Calibration Date: 08/22/2016
 3um 9um

0th Coef(*100): -16667 -22327
1st Coef(*100): 259190 132291
2nd Coef(*100): 3393 1374
H2O adj(mg/l*10k): 611 546

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:20
02 Std. Sol.	0.017	13:21
03 Room Air	0.000	13:21
04 Std. Sol.	0.017	13:22
05 Room Air	0.000	13:22
06 Std. Sol.	0.017	13:23
07 Room Air	0.000	13:24

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 201610A
County = 08 Oper No. = 666666

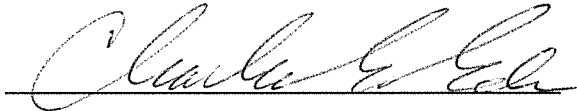

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:26
02 Std. Sol.	0.391	13:27
03 Room Air	0.000	13:27
04 Std. Sol.	0.394	13:28
05 Room Air	0.000	13:29
06 Std. Sol.	0.394	13:29
07 Room Air	0.000	13:30


08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16102

County = 08

Oper No. = 666666

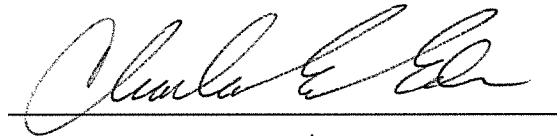

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:32

Test	AC	Time
01 Room Air	0.000	13:33
02 *Subject Test	INT*	13:33
03 Room Air	0.000	13:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

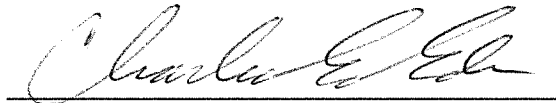

Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:35

Test	AC	Time
01 Diagnostic	OK	13:36
02 Room Air	0.000	13:36
03 *Subject Test	RFI*	13:36
04 Room Air	0.000	13:37

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 26
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:41
02 Std. Gas	0.082	13:42
03 Room Air	0.000	13:42
04 Std. Gas	0.082	13:43
05 Room Air	0.000	13:43
06 Std. Gas	0.082	13:43
07 Room Air	0.000	13:44

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

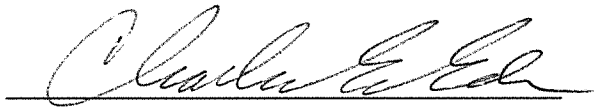

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:45
02 Std. Gas	0.082	13:45
03 Room Air	0.000	13:46
04 Std. Gas	0.082	13:46
05 Room Air	0.000	13:47
06 Std. Gas	0.082	13:47
07 Room Air	0.000	13:48

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

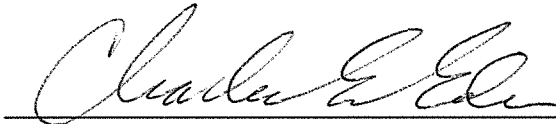

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = TOXL 8164.14.00 09/16
04/12/2017 13:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:48
02 Std. Gas	0.082	13:49
03 Room Air	0.000	13:49
04 Std. Gas	0.082	13:50
05 Room Air	0.000	13:50
06 Std. Gas	0.082	13:50
07 Room Air	0.000	13:51

Lot No = 17316080A2
Cyl No = 26
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

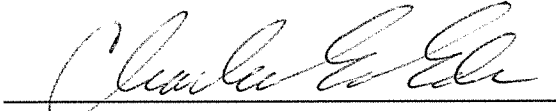

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC