

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006678

Location: TOXL

- |  | Check When Done |
|--|-----------------|
| A. Is the breath tube heated?  | <u>✓</u>        |
| B. Is the diagnostic check complete?   | <u>✓</u>        |
| C. Is the time, date, and year correct (re-set if necessary)?  | <u>✓</u>        |
| D. Print test. (Attach test record.)   | <u>✓</u>        |
| E. Low AC. Use $\leq 0.03$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>MP3003</u> Lot No. <u>201610A</u> Rep AC <u>0.020</u>  | <u>✓</u>        |
| F. High AC. Use $\geq 0.25$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>MP3063</u> Lot No. <u>16102</u> Rep AC <u>0.400</u>   | <u>✓</u>        |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.<br>(Attach test record.)<br>Sim SN <u>PR5135</u> Lot No. <u>JCS4</u> Solution <u>0.10AC + 0.05%</u> | <u>✓</u>        |
| H. RFI check. Run CMS mode. Key radio during CMS test.<br>(Attach test record.)  | <u>✓</u>        |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.) <u>0.080</u> AC  |                 |

Lot No. 17316080A2 Cyl No. 26 Exp. Date 8/5/2018

Test 1 0.082 Test 4 0.082 Test 7 0.082

Test 2 0.081 Test 5 0.081 Test 8 0.081

Test 3 0.081 Test 6 0.081 Test 9 0.082

Average 0.081

J. Tank pressure, Level 3 Func D Sub G

P: <sup>147</sup>~~175~~ <sup>CEE</sup> psi Reg 175 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4-12-2017  
Date

Charles E. Eder  
Field Inspector's Signature

12 Apr 17  
Date

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006678  
Location = TOXL                8164.14.00 09/16  
04/12/2017                      15:04

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            26  
Standard Expiration?       08/05/2018  
Oper No?                    666666

Flow Cal. Date:            08/22/2016  
Slope                      686  
Intercept                  -611596

IR Calibration Date:       08/22/2016  
                                  3um                    9um

-----  
0th Coef(\*100):        -15445                -21650  
1st Coef(\*100):        251854                131766  
2nd Coef(\*100):        3094                   1378  
H2O adj(mg/l\*10k):    549                    513

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:    PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006678  
Location = TOXL      8164.14.00 09/16  
04/12/2017      15:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:12
02 Std. Sol.	0.017	15:12
03 Room Air	0.000	15:13
04 Std. Sol.	0.017	15:14
05 Room Air	0.000	15:14
06 Std. Sol.	0.018	15:15
07 Room Air	0.000	15:15

08 Sim Temp = 34.0°C

Simul Ser No = MP3003  
Std Sol No = 201610A  
County = 08      Oper No. = 666666

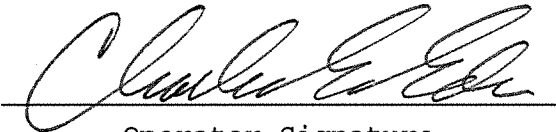

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006678  
Location = TOXL      8164.14.00 09/16  
04/12/2017      15:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:18
02 Std. Sol.	0.394	15:19
03 Room Air	0.000	15:19
04 Std. Sol.	0.396	15:20
05 Room Air	0.000	15:21
06 Std. Sol.	0.396	15:21
07 Room Air	0.000	15:22

08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16102

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.480 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006678  
Location = TOXL                    8164.14.00 09/16  
04/12/2017                            15:26

Test	AC	Time
01 Room Air	0.000	15:27
02 *Subject Test	INT*	15:27
03 Room Air	0.000	15:28

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 26  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10 AC + 0.05% ACETONE*  
Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006678  
Location = TOXL      8164.14.00 09/16  
04/12/2017      15:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:31
02 Std. Gas	0.082	15:32
03 Room Air	0.000	15:32
04 Std. Gas	0.081	15:32
05 Room Air	0.000	15:33
06 Std. Gas	0.081	15:33
07 Room Air	0.000	15:34

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006678  
Location = TOXL      8164.14.00 09/16  
04/12/2017      15:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:35
02 Std. Gas	0.082	15:35
03 Room Air	0.000	15:36
04 Std. Gas	0.081	15:36
05 Room Air	0.000	15:36
06 Std. Gas	0.081	15:37
07 Room Air	0.000	15:37

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006678  
Location = TOXL      8164.14.00 09/16  
04/12/2017      15:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:38
02 Std. Gas	0.082	15:38
03 Room Air	0.000	15:39
04 Std. Gas	0.081	15:39
05 Room Air	0.000	15:40
06 Std. Gas	0.082	15:40
07 Room Air	0.000	15:41

Lot No = 17316080A2  
Cyl No = 26  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

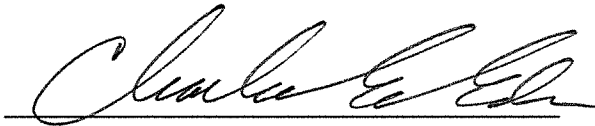

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC