

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006687

Location: TOXL

- Check When Done
- A. Is the breath tube heated? X
 - B. Is the diagnostic check complete? X
 - C. Is the time, date, and year correct (re-set if necessary)? X
 - D. Print test. (Attach test record.) X
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3846 Lot No. 16200 Rep AC 0.020 X
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR7111 Lot No. 16060 Rep AC 0.300 X
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN G10371 Lot No. 1CS4 Solution 0.10Ac + 0.05% X
Acetone
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) X
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC
Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18
Test 1 0.083 Test 4 0.083 Test 7 0.082
Test 2 0.081 Test 5 0.081 Test 8 0.082
Test 3 0.082 Test 6 0.082 Test 9 0.083
Average 0.082
 - J. Tank pressure, Level 3 Func D Sub G P: 989 psi Reg 990 psi
 - K. Remarks/Maintenance record: Annual inspection.

17 Apr 17
Date

17 Apr 17
Date

Roberta Mungen-Nimmo
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 08:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 09/06/2016
Slope 697
Intercept -689849

IR Calibration Date: 09/06/2016

	3um	9um
0th Coef(*100):	-14968	-21526
1st Coef(*100):	263834	133395
2nd Coef(*100):	2721	1501
H2O adj(mg/l*10k):	582	571

***** Printer Test End *****

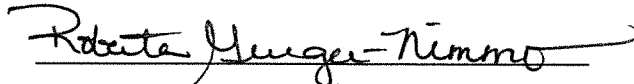
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 08:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:57
02 Std. Sol.	0.022	08:58
03 Room Air	0.000	08:58
04 Std. Sol.	0.022	08:59
05 Room Air	0.000	08:59
06 Std. Sol.	0.022	09:00
07 Room Air	0.000	09:01


08 Sim Temp = 34.0°C

Simul Ser No = DR3846

Std Sol No = 16200

County = 08

Oper No. = 888888

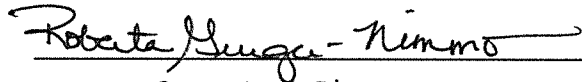

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

LOW AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:03
02 Std. Sol.	0.301	09:04
03 Room Air	0.000	09:05
04 Std. Sol.	0.302	09:06
05 Room Air	0.000	09:06
06 Std. Sol.	0.303	09:07
07 Room Air	0.000	09:07


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 16060

County = 08

Oper No. = 888888

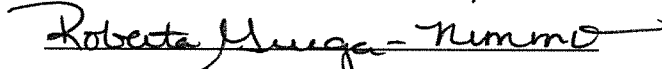

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

High AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:09

Test	AC	Time
01 Room Air	0.000	09:10
02 *Subject Test	INT*	09:11
03 Room Air	0.000	09:12

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

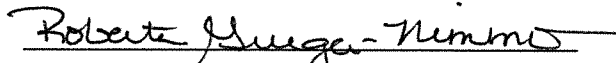

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:13

Test	AC	Time
01 Diagnostic	OK	09:13
02 Room Air	RFI*	09:14
03 Room Air	0.000	09:14

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

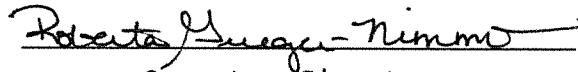

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:15
02 Std. Gas	0.083	09:16
03 Room Air	0.000	09:16
04 Std. Gas	0.081	09:17
05 Room Air	0.000	09:17
06 Std. Gas	0.082	09:17
07 Room Air	0.000	09:18

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

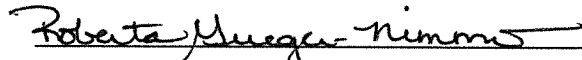
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:19
02 Std. Gas	0.083	09:19
03 Room Air	0.000	09:20
04 Std. Gas	0.081	09:20
05 Room Air	0.000	09:20
06 Std. Gas	0.082	09:21
07 Room Air	0.000	09:21

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

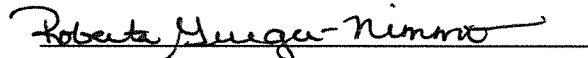
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = TOXL 8164.14.00 09/16
04/17/2017 09:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:22
02 Std. Gas	0.082	09:22
03 Room Air	0.000	09:23
04 Std. Gas	0.082	09:23
05 Room Air	0.000	09:24
06 Std. Gas	0.083	09:24
07 Room Air	0.000	09:25

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888

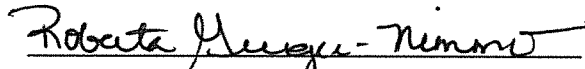
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000