



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (10-2016)

Serial Number	Instrument Location
80-006507	McKenzie County, LEC
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i> #210-1	Date 4-17-17
---	--------------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date 18 Apr 17
---	--------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006507  
Location = WATF      8164.13.00 06/09  
04/17/2017      16:20

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

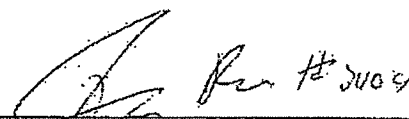
Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      09216080A3  
Standard Cyl #?      29  
Standard Expiration?      05/05/2018  
Oper No?      120701

Flow Cal. Date:      08/20/2015  
Slope      659  
Intercept      -658315

IR Calibration Date:      08/20/2015

	3um	9um
0th Coef(*100):	-9452	-11722
1st Coef(*100):	258808	139588
2nd Coef(*100):	4476	1674
H2O adj(mg/l*10k):	499	376

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
DANIEL BLOOD

Remarks: *Good Test*

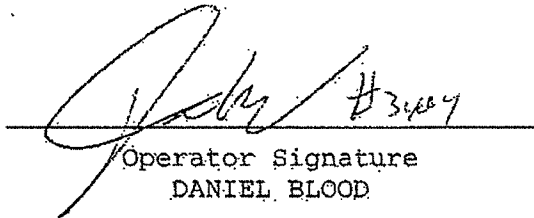
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006507  
Location = WATF      8164.13.00 06/09  
04/17/2017      16:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:22
02 Std. Gas	0.082	16:22
03 Room Air	0.000	16:23
04 Std. Gas	0.082	16:23
05 Room Air	0.000	16:23
06 Std. Gas	0.082	16:24
07 Room Air	0.000	16:24

Lot No = 09216080A3  
Cyl No = 29  
Exp Date = 05/05/2018  
County = 27      Oper No. = 120701

 #3464  
Operator Signature  
DANIEL BLOOD

Remarks: Good Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

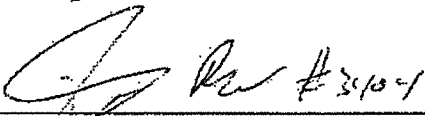
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006507  
Location = WATF      8164.13.00 06/09  
04/17/2017      16:25

Test	AC	Time
01 Diagnostic	OK	16:26
02 Room Air	RFI*	16:26
03 Room Air	0.000	16:27

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST; TEST TEST  
Sub DOB = 04/16/2017  
Sub Sex = Male      Weight = 100  
Test = DUI      Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 09216080A3  
Cyl No = 29  
Expiration Date = 05/05/2018  
County = 27      Oper No. = 120701

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
\_\_\_\_\_  
Operator Signature  
DANIEL BLOOD

Remarks: