

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006680

Location: TOX L

- |  | Check When Done |
|--|-----------------|
| A. Is the breath tube heated?  | <u>✓</u>        |
| B. Is the diagnostic check complete?   | <u>✓</u>        |
| C. Is the time, date, and year correct (re-set if necessary)?  | <u>✓</u>        |
| D. Print test. (Attach test record.)   | <u>✓</u>        |
| E. Low AC. Use $\leq 0.03$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>MP3003</u> Lot No. <u>201610A</u> Rep AC <u>0.020</u>  | <u>✓</u>        |
| F. High AC. Use $\geq 0.25$ AC in ACA mode. (Attach test record.)<br>Sim SN <u>MP3063</u> Lot No. <u>16102</u> Rep AC <u>0.400</u>   | <u>✓</u>        |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.<br>(Attach test record.)<br>Sim SN <u>DR5135</u> Lot No. <u>DCS4</u> Solution <u>0.10AC + 0.05% ACETONE</u> <del>ACETONE</del> <u>CEE</u> | <u>✓</u>        |
| H. RFI check. Run CMS mode. Key radio during CMS test.<br>(Attach test record.)  | <u>✓</u>        |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.<br>(Attach test record.) <u>0.080</u> AC  |                 |

Lot No. <u>17316080A2</u>	Cyl No. <u>28</u>	Exp. Date <u>8/5/2018</u>
Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.082</u>	Test 8 <u>0.081</u> <del>0.082</del> <u>CEE</u>
Test 3 <u>0.081</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

J. Tank pressure, Level 3 Func D Sub G P: 999 psi Reg 1000 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4-13-17

Date

Charles E. Ede  
Field Inspector's Signature

18 Apr 17

Date

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006680  
Location = TOXL                    8164.14.00 09/16  
04/13/2017                            16:21

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            28  
Standard Expiration?       08/05/2018  
Oper No?                    666666

Flow Cal. Date:            09/06/2016  
Slope                      681  
Intercept                  -620591

IR Calibration Date:       09/06/2016

	3um	9um
0th Coef(*100):	-14265	-21271
1st Coef(*100):	260743	129687
2nd Coef(*100):	3906	1392
H2O adj(mg/l*10k):	742	684

\*\*\*\*\* Printer Test End \*\*\*\*\*

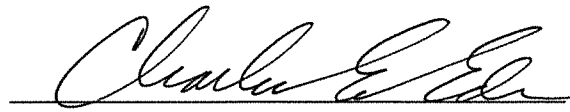

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006680  
Location = TOXL                8164.14.00 09/16  
04/13/2017                      16:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:22
02 Std. Sol.	0.018	16:23
03 Room Air	0.000	16:23
04 Std. Sol.	0.019	16:24
05 Room Air	0.000	16:25
06 Std. Sol.	0.019	16:25
07 Room Air	0.000	16:26

08 Sim Temp = 34.0°C

Simul Ser No = MP3003  
Std Sol No = 201610A  
County = 08                      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *LOW AC*  
                    *0.020 AC*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006680  
Location = TOXL                    8164.14.00 09/16  
04/13/2017                            16:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Sol.	0.395	16:28
03 Room Air	0.000	16:29
04 Std. Sol.	0.395	16:29
05 Room Air	0.000	16:30
06 Std. Sol.	0.394	16:31
07 Room Air	0.000	16:31


08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16102

County = 08

Oper No. = 666666

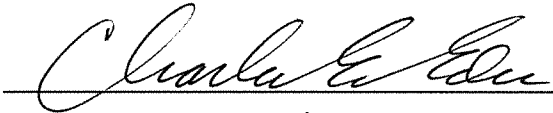

Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.400AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006680  
Location = TOXL                    8164.14.00 09/16  
04/13/2017                            16:33

Test	AC	Time
01 Room Air	0.000	16:34
02 *Subject Test	INT*	16:34
03 Room Air	0.000	16:35

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10 AC + 0.05% ACETONE*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006680  
Location = TOXL                    8164.14.00 09/16  
04/13/2017                            16:35

Test	AC	Time
01 Diagnostic	OK	16:36
02 Room Air	0.000	16:37
03 *Subject Test	RFI*	16:37
04 Room Air	0.000	16:37

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = TOXL      8164.14.00 09/16  
04/13/2017      16:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:38
02 Std. Gas	0.082	16:38
03 Room Air	0.000	16:39
04 Std. Gas	0.081	16:39
05 Room Air	0.000	16:40
06 Std. Gas	0.081	16:40
07 Room Air	0.000	16:41

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = TOXL      8164.14.00 09/16  
04/13/2017      16:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:41
02 Std. Gas	0.082	16:42
03 Room Air	0.000	16:42
04 Std. Gas	0.082	16:43
05 Room Air	0.000	16:43
06 Std. Gas	0.082	16:44
07 Room Air	0.000	16:44

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

  
\_\_\_\_\_  
Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = TOXL      8164.14.00 09/16  
04/13/2017      16:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:45
02 Std. Gas	0.081	16:45
03 Room Air	0.000	16:46
04 Std. Gas	0.081	16:46
05 Room Air	0.000	16:47
06 Std. Gas	0.082	16:47
07 Room Air	0.000	16:48

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC