

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006681

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR 5113 Lot No. 201507F Rep AC 0.020 ✓
- F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR 3378 Lot No. 16060 Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR 5135 Lot No. ICS 4 Solution 0.10AC + 0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 1002 psi Reg 1000 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4-13-17  
Date

18 Apr 17  
Date

Charles E. Ed  
Field Inspector's Signature

Deb Shanaver  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006681  
Location = TOXL                    8164.14.00 09/16  
04/13/2017                            15:12

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            28  
Standard Expiration?        08/05/2018  
Oper No?                    666666

Flow Cal. Date:            09/09/2016  
Slope                      694  
Intercept                  -677594

IR Calibration Date:        09/09/2016

	3um	9um
0th Coef(*100):	-10880	-18848
1st Coef(*100):	255177	129254
2nd Coef(*100):	4336	1756
H2O adj(mg/l*10k):	468	501

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:15
02 Std. Sol.	0.020	15:16
03 Room Air	0.000	15:16
04 Std. Sol.	0.020	15:17
05 Room Air	0.000	15:17
06 Std. Sol.	0.021	15:18
07 Room Air	0.000	15:19

08 Sim Temp = 34.0°C

Simul Ser No = DR5113  
Std Sol No = 201507F  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:21
02 Std. Sol.	0.303	15:22
03 Room Air	0.000	15:22
04 Std. Sol.	0.304	15:23
05 Room Air	0.000	15:24
06 Std. Sol.	0.305	15:24
07 Room Air	0.000	15:25

08 Sim Temp = 34.0°C

Simul Ser No = DR3378  
Std Sol No = 16060  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:27

Test	AC	Time
01 Room Air	0.000	15:28
02 *Subject Test	INT*	15:29
03 Room Air	0.000	15:29

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*

*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006681  
Location = TOXL                8164.14.00 09/16  
04/13/2017                        15:30

Test	AC	Time
01 Diagnostic	OK	15:31
02 Room Air	0.000	15:32
03 *Subject Test	RFI*	15:32
04 Room Air	0.000	15:32

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 17316080A2  
Cyl No = 28  
Expiration Date = 08/05/2018  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:33
02 Std. Gas	0.081	15:33
03 Room Air	0.000	15:34
04 Std. Gas	0.080	15:34
05 Room Air	0.000	15:35
06 Std. Gas	0.080	15:35
07 Room Air	0.000	15:35

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:37
02 Std. Gas	0.081	15:38
03 Room Air	0.000	15:38
04 Std. Gas	0.081	15:39
05 Room Air	0.000	15:39
06 Std. Gas	0.081	15:40
07 Room Air	0.000	15:40

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = TOXL      8164.14.00 09/16  
04/13/2017      15:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:47
02 Std. Gas	0.080	15:48
03 Room Air	0.000	15:48
04 Std. Gas	0.081	15:49
05 Room Air	0.000	15:49
06 Std. Gas	0.081	15:49
07 Room Air	0.000	15:50

Lot No = 17316080A2  
Cyl No = 28  
Exp Date = 08/05/2018  
County = 08      Oper No. = 666666

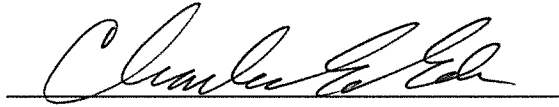

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000