

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006686 Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3003 Lot No. 201610A Rep AC 0.020 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 16102 Rep AC 0.400 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. IC54 Solution 0.10AC+0.05% ACETONE ✓
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 28 Exp. Date 8/5/2018

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 1008 psi Reg 1000 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

4-18-17
Date

18 Apr 17
Date

Charles E. Ede
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:13

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 09/07/2016
Slope 770
Intercept -749410

IR Calibration Date: 09/06/2016
 3um 9um

0th Coef(*100): -14519 -24061
1st Coef(*100): 252211 131592
2nd Coef(*100): 3527 1490
H2O adj(mg/l*10k): 522 594

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:18
02 Std. Sol.	0.018	10:19
03 Room Air	0.000	10:19
04 Std. Sol.	0.018	10:20
05 Room Air	0.000	10:20
06 Std. Sol.	0.019	10:21
07 Room Air	0.000	10:21

08 Sim Temp = 34.0°C

Simul Ser No = MP3003

Std Sol No = 201610A

County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:29
02 Std. Sol.	0.391	10:29
03 Room Air	0.000	10:30
04 Std. Sol.	0.391	10:31
05 Room Air	0.000	10:31
06 Std. Sol.	0.392	10:32
07 Room Air	0.000	10:33

08 Sim Temp = 34.0°C

Simul Ser No = MP3063

Std Sol No = 16102

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

1764 AC

0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:35

Test	AC	Time
01 Room Air	0.000	10:36
02 *Subject Test	INT*	10:36
03 Room Air	0.000	10:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

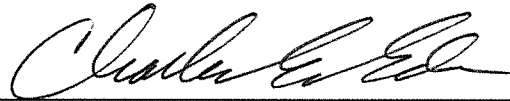

Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:37

Test	AC	Time
01 Diagnostic	OK	10:38
02 Room Air	0.000	10:38
03 *Subject Test	RFI*	10:39
04 Room Air	0.000	10:39

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:40
02 Std. Gas	0.081	10:40
03 Room Air	0.000	10:41
04 Std. Gas	0.081	10:41
05 Room Air	0.000	10:42
06 Std. Gas	0.081	10:42
07 Room Air	0.000	10:43

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

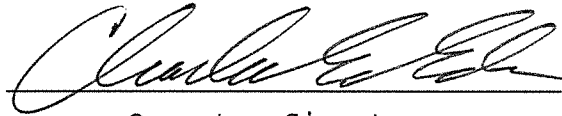

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = TOXL 8164.14.00 09/16
04/18/2017 10:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:44
02 Std. Gas	0.081	10:44
03 Room Air	0.000	10:44
04 Std. Gas	0.081	10:45
05 Room Air	0.000	10:45
06 Std. Gas	0.081	10:46
07 Room Air	0.000	10:46

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 08 Oper No. = 666666

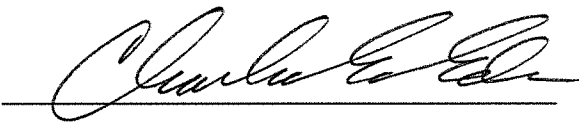

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

