



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number <i>80-006669</i>	Instrument Location <i>Washburn</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Richard Johnson</i>	Date <i>5/9/2017</i>
-----------------------------------------------------	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date <i>10 May 17</i>
----------------------------------------------	------------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006669
Location = WASH 8164.14.00 09/16
05/09/2017 12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:06
02 Std. Gas	0.080	12:07
03 Room Air	0.000	12:07
04 Std. Gas	0.080	12:07
05 Room Air	0.000	12:08
06 Std. Gas	0.080	12:08
07 Room Air	0.000	12:09

Lot No = 09216080A3
Cyl No = 27
Exp Date = 05/05/2018
County = 28 Oper No. = 118707

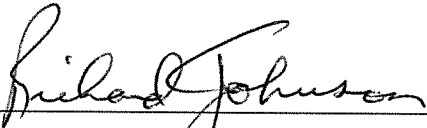
Operator Signature
RICHARD JOHNSON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006669
Location = WASH 8164.14.00 09/16
05/09/2017 12:10

Test	AC	Time
01 Room Air	RFI*	12:11
02 Room Air	0.000	12:12

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST T
Sub DOB = 01/01/2000
Sub Sex = Male Weight = 200
Test = OTH Cit = 1234
Dr. Lic. = ND/1234
Lot No = 09216080A3
Cyl No = 27
Expiration Date = 05/05/2018
County = 28 Oper No. = 118707

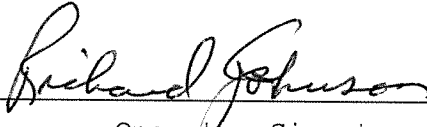

Operator Signature
RICHARD JOHNSON

Remarks: