

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006508

Location: Wahpeton jail

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>DR5113</u> Lot No. <u>201507F</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>DR3378</u> Lot No. <u>16060</u> Rep AC <u>0.300</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>DR3803</u> Lot No. <u>ICS4</u> Solution <u>0.10AC + 0.05% acetone</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080</u> AC | |

Lot No. <u>01316080A2</u>	Cyl No. <u>34</u>	Exp. Date <u>02/05/2018</u>
Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>
Average <u>0.080</u>		

J. Tank pressure, Level 3 Func D Sub G P: 868 psi Reg 870 psi

K. Remarks/Maintenance record: Annual inspection

08 May 17
Date
10 MAY 17
Date

Deb Shanaver
Field Inspector's Signature
Charles E. Eder
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 14:55

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01316080A2
Standard Cyl #? 34
Standard Expiration? 02/05/2018
Oper No? 777777

Flow Cal. Date: 08/20/2015
Slope 644
Intercept -568181

IR Calibration Date: 08/20/2015
 3um 9um

0th Coef(*100): -22365 -18627
1st Coef(*100): 256969 136095
2nd Coef(*100): 3168 1444
H2O adj(mg/l*10k): 693 453

***** Printer Test End *****

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 15:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:23
02 Std. Sol.	0.018	15:24
03 Room Air	0.000	15:25
04 Std. Sol.	0.018	15:26
05 Room Air	0.000	15:26
06 Std. Sol.	0.019	15:27
07 Room Air	0.000	15:27

08 Sim Temp = 34.0°C

Simul Ser No = DR5113
Std Sol No = 201507F
County = 39 Oper No. = 777777



Operator Signature
DEB SHANAVER

Remarks:

low AC 0.020AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 15:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:16
02 Std. Sol.	0.296	15:17
03 Room Air	0.000	15:18
04 Std. Sol.	0.297	15:18
05 Room Air	0.000	15:19
06 Std. Sol.	0.298	15:20
07 Room Air	0.000	15:20

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 39

Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

High AC 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 15:09

Test	AC	Time
01 Room Air	0.000	15:09
02 *Subject Test	INT*	15:10
03 Room Air	0.000	15:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 01316080A2
Cyl No = 34
Expiration Date = 02/05/2018
County = 39 Oper No. = 777777

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

Interferent detected check

Form 106-I8000

*0.10Ac + 0.05⁶²⁰
acetone*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 14:55

Test	AC	Time
01 Diagnostic	OK	14:56
02 Room Air	RFI*	14:56
03 Room Air	0.000	14:57

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 01316080A2
Cyl No = 34
Expiration Date = 02/05/2018
County = 39 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Shanaver

Operator Signature
DEB SHANAVER

Remarks:

RFI test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 14:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:58
02 Std. Gas	0.079	14:58
03 Room Air	0.000	14:59
04 Std. Gas	0.080	14:59
05 Room Air	0.000	15:00
06 Std. Gas	0.080	15:00
07 Room Air	0.000	15:01

Lot No = 01316080A2
Cyl No = 34
Exp Date = 02/05/2018
County = 39 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:02
02 Std. Gas	0.080	15:02
03 Room Air	0.000	15:03
04 Std. Gas	0.079	15:03
05 Room Air	0.000	15:04
06 Std. Gas	0.080	15:04
07 Room Air	0.000	15:05

Lot No = 01316080A2
Cyl No = 34
Exp Date = 02/05/2018
County = 39 Oper No. = 777777

Deb Shanaver
Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/08/2017 15:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:05
02 Std. Gas	0.080	15:06
03 Room Air	0.000	15:06
04 Std. Gas	0.080	15:07
05 Room Air	0.000	15:07
06 Std. Gas	0.079	15:08
07 Room Air	0.000	15:08

Lot No = 01316080A2
Cyl No = 34
Exp Date = 02/05/2018
County = 39 Oper No. = 777777


Operator Signature
DEB SHANAVER

Remarks:

Calibration check 0.080 AC