



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Kyle Johnson #664</i>		
Location <i>WFPD</i>	Intoxilyzer® Serial Number <i>80-006671</i>	
Gas Lot Number <i>01316080A2</i>	Gas Cylinder Number <i>35</i>	Gas Expiration Date <i>02/05/2018</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.080* AC
  - B. *0.080* AC
  - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>05/11/2017</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>11 May 17</i>

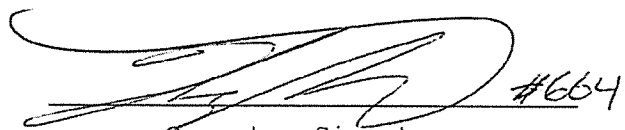
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006671  
Location = WFPD      8164.14.00 09/16  
05/11/2017      01:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:01
02 Std. Gas	0.080	01:02
03 Room Air	0.000	01:02
04 Std. Gas	0.080	01:02
05 Room Air	0.000	01:03
06 Std. Gas	0.081	01:03
07 Room Air	0.000	01:04

Lot No = 01316080A2  
Cyl No = 35  
Exp Date = 02/05/2018  
County = 09      Oper No. = 130781

  
Operator Signature  
KYLE JOHNSON

Remarks: