

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006511

Location: TOXL

- Check When Done
- A. Is the breath tube heated? X
 - B. Is the diagnostic check complete? X
 - C. Is the time, date, and year correct (re-set if necessary)? X
 - D. Print test. (Attach test record.) X
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3846 Lot No. 16200 Rep AC 0.020AC X
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR7111 Lot No. 16060 Rep AC 0.300AC X
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5135 Lot No. 1CS4 Solution 0.10AC + 0.05% X
Acetone
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) X
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 17316080A2 Cyl No. 27 Exp. Date 5 Aug 18
Test 1 0.082 Test 4 0.082 Test 7 0.082
Test 2 0.082 Test 5 0.082 Test 8 0.082
Test 3 0.081 Test 6 0.082 Test 9 0.082
Average 0.082

J. Tank pressure, Level 3 Func D Sub G P: 307 psi Reg 300 psi

K. Remarks/Maintenance record: Annual inspection; calibrated tank sensor.

12 May 17
Date

12 May 17
Date

Roberta Huegel-Timm
Field Inspector's Signature

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 10:56

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

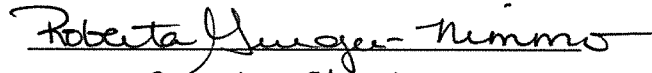
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 27
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 08/21/2015
Slope 688
Intercept -714727

IR Calibration Date: 08/21/2015

	3um	9um
0th Coef(*100):	-19744	-15946
1st Coef(*100):	272125	142950
2nd Coef(*100):	2975	1383
H2O adj(mg/l*10k):	725	394

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

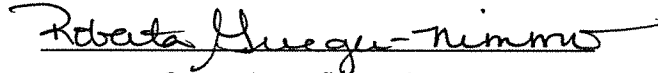
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 10:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:51
02 Std. Sol.	0.021	10:51
03 Room Air	0.000	10:52
04 Std. Sol.	0.021	10:53
05 Room Air	0.000	10:53
06 Std. Sol.	0.021	10:54
07 Room Air	0.000	10:55

08 Sim Temp = 34.0°C

Simul Ser No = DR3846
Std Sol No = 16200
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

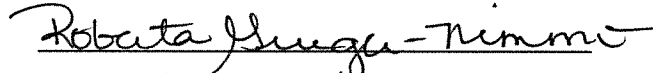
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 10:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Sol.	0.306	10:58
03 Room Air	0.000	10:59
04 Std. Sol.	0.305	10:59
05 Room Air	0.000	11:00
06 Std. Sol.	0.306	11:01
07 Room Air	0.000	11:01

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 16060
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

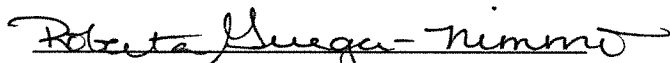
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 11:20

Test	AC	Time
01 Room Air	0.000	11:20
02 *Subject Test	INT*	11:21
03 Room Air	0.000	11:22

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

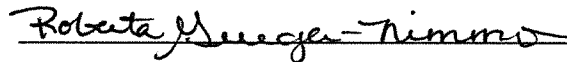
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 11:24

Test	AC	Time
01 Diagnostic	OK	11:25
02 Room Air	0.000	11:25
03 *Subject Test	RFI*	11:26
04 Room Air	0.000	11:26

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 27
Expiration Date = 08/05/2018
County = 08 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

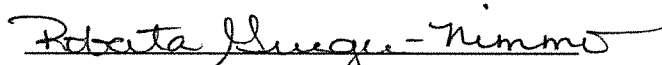
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 11:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:05
02 Std. Gas	0.082	11:05
03 Room Air	0.000	11:06
04 Std. Gas	0.082	11:06
05 Room Air	0.000	11:07
06 Std. Gas	0.081	11:07
07 Room Air	0.000	11:08

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

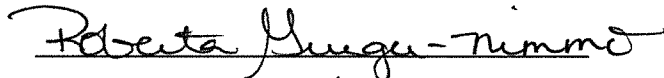
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 11:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:09
02 Std. Gas	0.082	11:09
03 Room Air	0.000	11:09
04 Std. Gas	0.082	11:10
05 Room Air	0.000	11:10
06 Std. Gas	0.082	11:11
07 Room Air	0.000	11:11

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

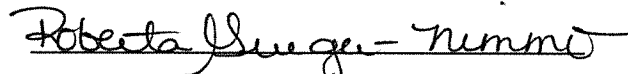
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = TOXL 8164.14.00 09/16
05/12/2017 11:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:12
02 Std. Gas	0.082	11:13
03 Room Air	0.000	11:13
04 Std. Gas	0.082	11:13
05 Room Air	0.000	11:14
06 Std. Gas	0.082	11:14
07 Room Air	0.000	11:15

Lot No = 17316080A2
Cyl No = 27
Exp Date = 08/05/2018
County = 08 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC
Form 106-I8000