



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number <i>80-006686</i>	Instrument Location <i>HP office - Walsh Co SO</i> <i>Deb Shanaver</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Barbara [unclear] #1442</i>	Date <i>5-17-2017</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>26 May 17</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = GRAF 8164.14.00 09/16
05/17/2017 18:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:18
02 Std. Gas	0.080	18:18
03 Room Air	0.000	18:19
04 Std. Gas	0.079	18:19
05 Room Air	0.000	18:20
06 Std. Gas	0.079	18:20
07 Room Air	0.000	18:21

Lot No = 17316080A2
Cyl No = 28
Exp Date = 08/05/2018
County = 50 Oper No. = 130890

 #1442

Operator Signature
BRADLEY WINGERTER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = GRAF 8164.14.00 09/16
05/17/2017 18:23

Test	AC	Time
01 Room Air	RFI*	18:25
02 Room Air	0.000	18:26

*Invalid Test
Inhibited - RFI

Sub Name = DOE, JANE NA
Sub DOB = 12/12/1972
Sub Sex = Female Weight = 135
Test = DUI Cit = 1234567
Dr. Lic. = ND/DOE724534
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 50 Oper No. = 130890

 #1442

Operator Signature
BRADLEY WINGERTER

Remarks:

Form 106-I8000