



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

| | |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------|
| Serial Number <i>80-006de78</i> | Instrument Location <i>Hill</i> |
| Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____ | |

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|------------------------------------------------|------------------------|
| Field Inspector Signature <i>Raf Welles</i> | Date <i>5/18/17</i> |
|------------------------------------------------|------------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

| | |
|----------------------------------------------|------------------------------------|
| Reviewed/Certified by <i>Deb Shenaver</i> | Certified Date <i>22 May 17</i> |
|----------------------------------------------|------------------------------------|

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
05/18/2017 20:24

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19615080A5
Standard Cyl #? 32
Standard Expiration? 09/05/2017
Oper No? 058405

Flow Cal. Date: 08/22/2016
Slope 686
Intercept -611596

IR Calibration Date: 08/22/2016
 3um 9um

0th Coef(*100): -15445 -21650
1st Coef(*100): 251854 131766
2nd Coef(*100): 3094 1378
H2O adj(mg/l*10k): 549 513

***** Printer Test End *****



Operator Signature
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
05/18/2017 20:33

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 20:34 |
| 02 Std. Gas | 0.079 | 20:34 |
| 03 Room Air | 0.000 | 20:35 |
| 04 Std. Gas | 0.080 | 20:35 |
| 05 Room Air | 0.000 | 20:36 |
| 06 Std. Gas | 0.080 | 20:36 |
| 07 Room Air | 0.000 | 20:37 |

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405


Operator Signature
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
05/18/2017 20:39

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 20:40 |
| 02 Std. Gas | RFI* | 20:41 |
| 03 Room Air | 0.000 | 20:41 |

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405



Operator Signature
RAY WEBER

Remarks: