

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006507

Location: WATF

(MCKENZIE COUNTY
SD.
MCKENZIE COUNTY L.E.C.)

- Check When Done
- A. Is the breath tube heated? ✓
 - B. Is the diagnostic check complete? ✓
 - C. Is the time, date, and year correct (re-set if necessary)? ✓
 - D. Print test. (Attach test record.) ✓
 - E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN DR3803^{CEE} Lot No. 201507F Rep AC 0.020
DR5113
 - F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN DR3378 Lot No. 16060 Rep AC 0.300
 - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN DR5113^{CEE} Lot No. ICS4 Solution 0.10 AC + 0.05%
ACETONE
DR3803
 - H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
 - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. <u>09216080A3^{CEE}</u>	Cyl No. <u>29</u>	Exp. Date <u>5/5/2018</u>
Test 1 <u>0.0801</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

- J. Tank pressure, Level 3 Func D Sub G P: 775 psi Reg 750 psi
- K. Remarks/Maintenance record: ANNUAL INSPECTION

5/16/2017
Date

Charles E. Ed
Field Inspector's Signature

22 May 17
Date

Deb Shanaver
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 09216080A3
Standard Cyl #? 29
Standard Expiration? 05/05/2018
Oper No? 666666

Flow Cal. Date: 08/20/2015
Slope 659
Intercept -658315

IR Calibration Date: 08/20/2015
 3um 9um

0th Coef(*100): -9452 -11722
1st Coef(*100): 258808 139588
2nd Coef(*100): 4476 1674
H2O adj(mg/l*10k): 499 376

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:34
02 Std. Sol.	0.020	12:35
03 Room Air	0.000	12:36
04 Std. Sol.	0.020	12:36
05 Room Air	0.000	12:37
06 Std. Sol.	0.020	12:38
07 Room Air	0.000	12:38

08 Sim Temp = 34.0°C

Simul Ser No = DR5113
Std Sol No = 201507F
County = 27 Oper No. = 666666

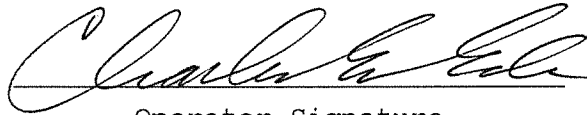

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:45
02 Std. Sol.	0.308	12:45
03 Room Air	0.000	12:46
04 Std. Sol.	0.306	12:47
05 Room Air	0.000	12:47
06 Std. Sol.	0.306	12:48
07 Room Air	0.000	12:49

08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 16060

County = 27

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:49

Test	AC	Time
01 Room Air	0.000	12:50
02 *Subject Test	INT*	12:51
03 Room Air	0.000	12:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 29
Expiration Date = 05/05/2018
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent*
0.10A C + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:52

Test	AC	Time
01 Diagnostic	OK	12:53
02 Room Air	0.000	12:54
03 *Subject Test	RFI*	12:54
04 Room Air	0.000	12:54

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 09216080A3
Cyl No = 29
Expiration Date = 05/05/2018
County = 27 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:56
02 Std. Gas	0.081	12:56
03 Room Air	0.000	12:57
04 Std. Gas	0.080	12:57
05 Room Air	0.000	12:58
06 Std. Gas	0.080	12:58
07 Room Air	0.000	12:58

Lot No = 09216080A3
Cyl No = 29
Exp Date = 05/05/2018
County = 27 Oper No. = 666666

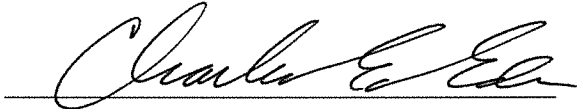

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 12:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:59
02 Std. Gas	0.081	13:00
03 Room Air	0.000	13:00
04 Std. Gas	0.081	13:00
05 Room Air	0.000	13:01
06 Std. Gas	0.080	13:01
07 Room Air	0.000	13:02

Lot No = 09216080A3
Cyl No = 29
Exp Date = 05/05/2018
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
05/16/2017 13:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:03
02 Std. Gas	0.081	13:03
03 Room Air	0.000	13:03
04 Std. Gas	0.080	13:04
05 Room Air	0.000	13:04
06 Std. Gas	0.080	13:05
07 Room Air	0.000	13:05

Lot No = 09216080A3
Cyl No = 29
Exp Date = 05/05/2018
County = 27 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC