

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006673

A. GENERAL Setup and Checks:

- Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
- Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
- Barcode Reader (Level 1,B):
 - Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
- Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-006673 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12401 s: 1 C: 47.00
9: 13197 s: 2 B: 45 0
(RFI indicator with keyed radio): 1
- Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
- Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 940 CF: 1.07
- Flow Monitor (Level 3,D,F): Flow Sensor, R: 252 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 48.5 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 688 psi P (gauge): 700 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP3064 Lot #: MILLI Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP3003 Lot #: 16200 Exp date: 8-17-18
Known value: 0.020 AC
Results: 0.019 AC 0.019 AC 0.020 AC
3. High AC (Level 1,C):
Sim ser #: MP3070 Lot #: 16060 Exp date: 2-10-18
Known value: 0.300 AC
Results: 0.292 AC 0.293 AC 0.293 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5135 Lot #: ICS4 Exp date: 9/9/2017
Known value: 0.10AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5143 Lot #: ACE 2 Exp date: 9/29/17
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR3377 Lot #: MET 2 Exp date: 9/29/17
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR3453 Lot #: IPA 2 Exp date: 9/29/17
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0811 Std Dev: 0.0003 Rel Std Dev: 0.38%

Maintenance/Repairs/Notes: _____

Instrument is acceptable to be used in the field.

Charles E. Edm
Field Inspector's Signature

5/25/2017
Date

Deb Shanover
Reviewed by

26 May 17
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 15:45

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuvw
ABCDEFGHIJKLMNQRSTUUVW

32 characters per line
abcdefghijklmnopqrstuvwxy123456
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuvwxy1234567890abcdef
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MenFull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 03/30/2017
Slope: 697
Intercept: -667725

IR Calibration Date: 03/30/2017
3um 9um

0th Coef(*100):	-9136	-17613
1st Coef(*100):	267126	133333
2nd Coef(*100):	5438	1698
H2O adj(mg/l)*10K):	684	589

***** Test End *****



Operator Signature
CHARLES EDER

Remarks: *Internal*
PRINT TEST

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 15:56

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 28
Standard Expiration? 08/05/2018
Oper No? 666666

Flow Cal. Date: 03/30/2017
Slope 697
Intercept -667725

IR Calibration Date: 03/30/2017

	3um	9um
0th Coef(*100):	-9136	-17613
1st Coef(*100):	267126	133333
2nd Coef(*100):	5438	1698
H2O adj(mg/l*10k):	684	589

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

EXTERNAL PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 15:57

Test	AC	Time
01 Room Air	0.000	15:58
02 *Subject Test	REF*	15:58
03 Room Air	0.000	15:59

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: 'R' KEY REFUSAL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 15:59

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *DIAGNOSTIC*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:08
02 Std. Sol.	0.000	16:09
03 Room Air	0.000	16:09
04 Std. Sol.	0.000	16:10
05 Room Air	0.000	16:10
06 Std. Sol.	0.000	16:11
07 Room Air	0.000	16:11

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666

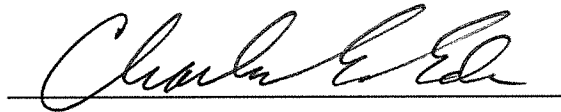

Operator Signature
CHARLES EDER

Remarks: ZERO AC
 0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:14
02 Std. Sol.	0.019	16:15
03 Room Air	0.000	16:15
04 Std. Sol.	0.019	16:16
05 Room Air	0.000	16:16
06 Std. Sol.	0.020	16:17
07 Room Air	0.000	16:18

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 16200
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:26
02 Std. Sol.	0.292	16:27
03 Room Air	0.000	16:28
04 Std. Sol.	0.293	16:28
05 Room Air	0.000	16:29
06 Std. Sol.	0.293	16:30
07 Room Air	0.000	16:30


08 Sim Temp = 34.0°C

Simul Ser No = MP3070

Std Sol No = 16060

County = 08

Oper No. = 666666

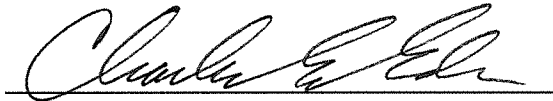

Operator Signature

CHARLES EDER

Remarks:

HIGH AC

0.300AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:36

Test	AC	Time
01 Room Air	0.000	16:37
02 *Subject Test	INT*	16:37
03 Room Air	0.000	16:38

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:38

Test	AC	Time
01 Room Air	0.000	16:39
02 *Subject Test	INT*	16:39
03 Room Air	0.000	16:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent
0.05% ACETONE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:41

Test	AC	Time
01 Room Air	0.000	16:42
02 *Subject Test	INT*	16:42
03 Room Air	0.000	16:43

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

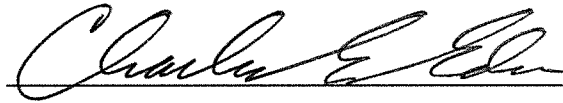

Operator Signature
CHARLES EDER

Remarks:

Interferent
0.10% METHANOL


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:44

Test	AC	Time
01 Room Air	0.000	16:45
02 *Subject Test	INT*	16:45
03 Room Air	0.000	16:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

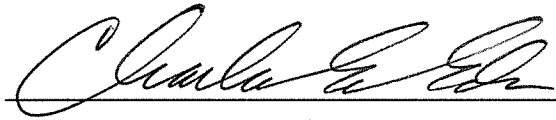

Operator Signature
CHARLES EDER

Remarks:

Interferent

0.10% ISOPROPANOL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/24/2017 16:48

Test	AC	Time
01 Room Air	AMB*	16:49
02 Room Air	PUR**	16:50

*Invalid Test
 Check Ambient Conditions
**Purge Fail

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*
WITH PURGE FAIL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 14:54

Test	AC	Time
01 Room Air	0.000	14:55
02 Invalid Sample	X.XXX	14:55
03 Room Air	0.000	14:56

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INVALID SAMPLE*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 14:56

Test	AC	Time
01 Room Air	0.000	14:57
02 *Subject Test	ABT*	14:57
03 Room Air	0.000	14:58

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

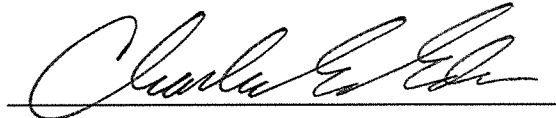

Operator Signature
CHARLES EDER

Remarks: *SEQUENCE ABORTED*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 14:58

Test	AC	Time
01 Room Air	0.000	14:59
02 Subject Test 1	0.014	14:59
03 Room Air	0.000	15:00
04 Reported AC	0.014	14:59

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *ABA MODE*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:01

Test	AC	Time
01 Diagnostic	OK	15:02
02 Room Air	0.000	15:02
03 Subject Test 1	0.009	15:03
04 Room Air	0.000	15:05
05 Std. Gas	0.090*	15:06
06 Room Air	0.000	15:06

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CAL CHK OUT TOL
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

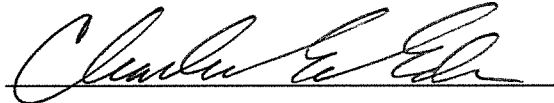

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:08

Test	AC	Time
01 Diagnostic	OK	15:09
02 Room Air	0.000	15:10
03 *Subject Test	RFI*	15:10
04 Room Air	0.000	15:10

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

RFI DETECTED

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:11

Test	AC	Time
01 Diagnostic	OK	15:12
02 Room Air	0.000	15:13
03 *Subject Test	0.000*	15:16
04 Room Air	0.000	15:18
05 Std. Gas	0.081	15:19
06 Room Air	0.000	15:20
07 *Subject Test	0.000*	15:23
08 Room Air	0.000	15:24

No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DEFICIENT SAMP
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DEFICIENT
SAMPLES

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:25

Test	AC	Time
01 Diagnostic	OK	15:26
02 Room Air	IPS*	15:26
03 Room Air	0.000	15:26

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *IMPROPER SAMPLE*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:27

Test	AC	Time
01 Diagnostic	OK	15:28
02 Room Air	0.000	15:29
03 Subject Test 1	0.392*	15:29
04 Room Air	0.000	15:32
05 Std. Gas	0.082	15:33
06 Room Air	0.000	15:34
07 Subject Test 2	0.298*	15:35
08 Room Air	0.000	15:35

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *DIFFERENCE TOO GREAT*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:37


Test	AC	Time
01 Diagnostic	OK	15:38
02 Room Air	0.000	15:38
03 Subject Test 1	0.136	15:38
04 Room Air	0.000	15:41
05 Std. Gas	0.081	15:42
06 Room Air	0.000	15:43
07 Subject Test 2	0.134	15:44
08 Room Air	0.000	15:45
09 Reported AC	0.134	15:44

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = CMS MODE
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 28
Expiration Date = 08/05/2018
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

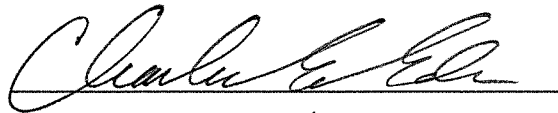

Operator Signature
CHARLES EDER

Remarks:

CMS MODE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/25/2017 15:49

Test	AC	Time
01 Room Air	0.000	15:49
02 Std. Gas	0.081	15:49
03 Room Air	0.000	15:50
04 Std. Gas	0.081	15:50
05 Room Air	0.000	15:51
06 Std. Gas	0.081	15:51
07 Room Air	0.000	15:52
08 Std. Gas	0.081	15:52
09 Room Air	0.000	15:53
10 Std. Gas	0.082	15:53
11 Room Air	0.000	15:53
12 Std. Gas	0.081	15:54
13 Room Air	0.000	15:54
14 Std. Gas	0.081	15:55
15 Room Air	0.000	15:55
16 Std. Gas	0.081	15:56
17 Room Air	0.000	15:56
18 Std. Gas	0.081	15:56
19 Room Air	0.000	15:57
20 Std. Gas	0.081	15:57
21 Room Air	0.000	15:58
22 Std. Gas	0.081	15:58
23 Room Air	0.000	15:59
24 Std. Gas	0.081	15:59
25 Room Air	0.000	16:00
26 Std. Gas	0.081	16:00
27 Room Air	0.000	16:00
28 Std. Gas	0.081	16:01
29 Room Air	0.000	16:01
30 Std. Gas	0.081	16:02
31 Room Air	0.000	16:02
32 Std. Gas	0.081	16:02
33 Room Air	0.000	16:03
34 Std. Gas	0.081	16:03
35 Room Air	0.000	16:04
36 Std. Gas	0.081	16:04
37 Room Air	0.000	16:05
38 Std. Gas	0.082	16:05
39 Room Air	0.000	16:06
40 Std. Gas	0.081	16:06
41 Room Air	0.000	16:06
42 Cal Check Stats:		
43 Average	0.0811	
44 Std Dev	0.0003	
45 Rel Std Dev	0.38%	



Operator Signature
CHARLES EDER

Remarks: LOT: 17316080A2
EXP: 8/5/2018

Form 106-I8000

CYL: 028

STABILITY TEST