

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006673

Location: TOXL

- | | Check When Done |
|--|-----------------|
| A. Is the breath tube heated? | <u>✓</u> |
| B. Is the diagnostic check complete? | <u>✓</u> |
| C. Is the time, date, and year correct (re-set if necessary)? | <u>✓</u> |
| D. Print test. (Attach test record.) | <u>✓</u> |
| E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3003</u> Lot No. <u>16200</u> Rep AC <u>0.020</u> | <u>✓</u> |
| F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN <u>MP3063</u> Lot No. <u>16102</u> Rep AC <u>0.400</u> | <u>✓</u> |
| G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.)
Sim SN <u>MP3071</u> Lot No. <u>ICS5</u> Solution <u>0.10AC + 0.05% ACETONE</u> | <u>✓</u> |
| H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) | <u>✓</u> |
| I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) <u>0.080AC</u> | |

Lot No. <u>03417080A3</u>	Cyl No. <u>25</u>	Exp. Date <u>3/5/2019</u>
Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

J. Tank pressure, Level 3 Func D Sub G P: 361 psi Reg 375 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

5/26/2017
Date

30 May 17
Date

Charles E. Ed
Field Inspector's Signature

Deb Shanover
Reviewed by

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-006673
 Location = TOXL 8164.14.00 09/16
 05/26/2017 11:16

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 03417080A3
 Standard Cyl #? 25
 Standard Expiration? 03/05/2019
 Oper No? 666666

Flow Cal. Date: 03/30/2017
 Slope 697
 Intercept -667725

IR Calibration Date: 03/30/2017

	3um	9um

0th Coef(*100):	-9136	-17613
1st Coef(*100):	267126	133333
2nd Coef(*100):	5438	1698
H2O adj(mg/l*10k):	684	589

***** Printer Test End *****



Operator Signature
 CHARLES EDER

Remarks: CEE
 PRINT TEST
 PRINT

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 10:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:51
02 Std. Sol.	0.019	10:52
03 Room Air	0.000	10:52
04 Std. Sol.	0.019	10:53
05 Room Air	0.000	10:53
06 Std. Sol.	0.019	10:54
07 Room Air	0.000	10:55

08 Sim Temp = 34.0°C

Simul Ser No = MP3003
Std Sol No = 16200
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:08

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:08
02 Std. Sol.	0.392	11:09
03 Room Air	0.000	11:10
04 Std. Sol.	0.392	11:10
05 Room Air	0.000	11:11
06 Std. Sol.	0.392	11:12
07 Room Air	0.000	11:12

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 16102
County = 08 Oper No. = 666666

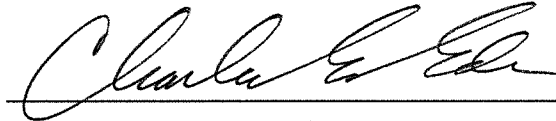

Operator Signature
CHARLES EDER

Remarks: H16A AC
 0.400 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:17

Test	AC	Time
01 Room Air	0.000	11:18
02 *Subject Test	INT*	11:18
03 Room Air	0.000	11:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 25
Expiration Date = 03/05/2019
County = 08 Oper No. = 666666

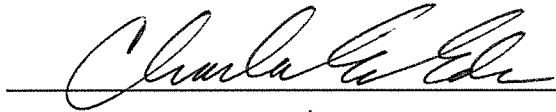

Operator Signature
CHARLES EDER

Remarks:

Interferent

0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:19

Test	AC	Time
01 Diagnostic	OK	11:20
02 Room Air	0.000	11:20
03 *Subject Test	RFI*	11:21
04 Room Air	0.000	11:21

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 25
Expiration Date = 03/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:22
02 Std. Gas	0.081	11:22
03 Room Air	0.000	11:23
04 Std. Gas	0.082	11:23
05 Room Air	0.000	11:24
06 Std. Gas	0.081	11:24
07 Room Air	0.000	11:25

Lot No = 03417080A3
Cyl No = 25
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.081	11:26
03 Room Air	0.000	11:26
04 Std. Gas	0.081	11:27
05 Room Air	0.000	11:27
06 Std. Gas	0.081	11:27
07 Room Air	0.000	11:28

Lot No = 03417080A3
Cyl No = 25
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/26/2017 11:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:29
02 Std. Gas	0.082	11:29
03 Room Air	0.000	11:30
04 Std. Gas	0.082	11:30
05 Room Air	0.000	11:31
06 Std. Gas	0.081	11:31
07 Room Air	0.000	11:31

Lot No = 03417080A3
Cyl No = 25
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000