



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (10-2016)

Serial Number <i>80-006678</i>	Instrument Location <i>Hill</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Ray Weber</i>	Date <i>6-2-17</i>
---	-----------------------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Deb Shanaver</i>	Certified Date <i>68 June 17</i>
--	-------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006678  
Location = HILL                8164.14.00 09/16  
06/02/2017                      01:18

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19615080A5  
Standard Cyl #?            32  
Standard Expiration?       09/05/2017  
Oper No?                    058405

Flow Cal. Date:            08/22/2016  
Slope                      686  
Intercept                  -611596

IR Calibration Date:        08/22/2016  
                                    3um                      9um

-----  
0th Coef(\*100):            -15445                      -21650  
1st Coef(\*100):            251854                      131766  
2nd Coef(\*100):            3094                        1378  
H2O adj(mg/l\*10k):        549                         513

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
\_\_\_\_\_  
Operator Signature  
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006678  
Location = HILL                    8164.14.00 09/16  
06/02/2017                            01:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:15
02 Std. Gas	0.079	01:15
03 Room Air	0.000	01:15
04 Std. Gas	0.080	01:16
05 Room Air	0.000	01:16
06 Std. Gas	0.080	01:17
07 Room Air	0.000	01:17

Lot No = 19615080A5  
Cyl No = 32  
Exp Date = 09/05/2017  
County = 49                            Oper No. = 058405

  
\_\_\_\_\_  
Operator Signature  
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006678  
Location = HILL                    8164.14.00 09/16  
06/02/2017                            01:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:20
02 Std. Gas	0.080	01:20
03 Room Air	RFI*	01:21
04 Room Air	0.000	01:21

\*Invalid Test  
Inhibited - RFI

Lot No = 19615080A5  
Cyl No = 32  
Exp Date = 09/05/2017  
County = 49                            Oper No. = 058405

  
\_\_\_\_\_  
Operator Signature  
RAY WEBER

Remarks: