



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number 80-006677	Instrument Location WARD COUNTY JAIL
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i> #002	Date 6-7-17
--	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by Debs Shehaver	Certified Date 08 June 17
---	-------------------------------------

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-006677
 Location = WARD 8164.14.00 09/16
 06/07/2017 08:14

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 09216080A3
 Standard Cyl #? 32
 Standard Expiration? 05/05/2018
 Oper No? 039709

Flow Cal. Date: 08/24/2016
 Slope 696
 Intercept -655873

IR Calibration Date: 08/24/2016
 3um 9um

	3um	9um
0th Coef(*100):	-12424	-22862
1st Coef(*100):	263434	131353
2nd Coef(*100):	5070	1477
H2O adj(mg/l*10k):	569	590

***** Printer Test End *****

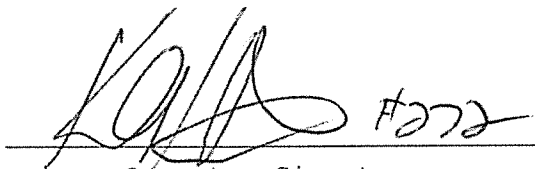
 Operator Signature
 KEVIN HUSTON

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006677
Location = WARD 8164.14.00 09/16
06/07/2017 08:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:21
02 Std. Gas	0.080	08:21
03 Room Air	0.000	08:21
04 Std. Gas	0.081	08:22
05 Room Air	0.000	08:22
06 Std. Gas	0.081	08:23
07 Room Air	0.000	08:23

Lot No = 09216080A3
Cyl No = 32
Exp Date = 05/05/2018
County = 51 Oper No. = 039709

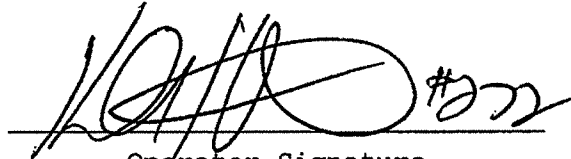

Operator Signature
KEVIN HUSTON

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006677
Location = WARD 8164.14.00 09/16
06/07/2017 08:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	08:18
02 Room Air	0.000	08:18

*Invalid Test
Inhibited - RFI

Lot No = 09216080A3
Cyl No = 32
Exp Date = 05/05/2018
County = 51 Oper No. = 039709



Operator Signature
KEVIN HUSTON

Remarks: