



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number <i>80-006682</i>	Instrument Location <i>BCCC</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

*CEE
NEW
LOCATION* →

Field Inspector Signature <i>[Signature]</i>	Date <i>6/28/17</i>
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>[Signature]</i>	Certified Date <i>6/28/2017</i>
---	------------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = BCCC 8164.14.00 09/16
06/28/2017 03:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	03:30
02 Std. Gas	0.080	03:31
03 Room Air	0.000	03:31
04 Std. Gas	0.080	03:32
05 Room Air	0.000	03:32
06 Std. Gas	0.080	03:32
07 Room Air	0.000	03:33

Lot No = 03417080A3
Cyl No = 25
Exp Date = 03/05/2019
County = 02 Oper No. = 100012



Operator Signature
DANA RUSTEBAKKE

Remarks:

Form 106-I8000

ACA Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = BCCC 8164.14.00 09/16
06/28/2017 03:35

Test	AC	Time
01 Room Air	0.000	03:36
02 *Subject Test	RFI*	03:36
03 Room Air	0.000	03:37

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 01/01/1990
Sub Sex = Male Weight = 200
Test = DUI Cit = 12345
Dr. Lic. = ND/12345
Lot No = 03417080A3
Cyl No = 25
Expiration Date = 03/05/2019
County = 02 Oper No. = 100012



Operator Signature
DANA RUSTEBAKKE

Remarks:

RFI Test

Form 106-I8000