

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006672

Location: TOXL

- Check When Done
- A. Is the breath tube heated? ✓
- B. Is the diagnostic check complete? ✓
- C. Is the time, date, and year correct (re-set if necessary)? ✓
- D. Print test. (Attach test record.) ✓
- E. Low AC. Use ≤ 0.03 AC in ACA mode. (Attach test record.)
Sim SN MP3068 Lot No. 201603H Rep AC 0.010 ✓
- F. High AC. Use ≥ 0.25 AC in ACA mode. (Attach test record.)
Sim SN MP3063 Lot No. 201603A Rep AC 0.300 ✓
- G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.
(Attach test record.) ✓
Sim SN DR3451 Lot No. ICS4 Solution 0.10AC + 0.05% ACETONE
- H. RFI check. Run CMS mode. Key radio during CMS test.
(Attach test record.) ✓
- I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.
(Attach test record.) 0.080 AC

Lot No. 03417080A3 Cyl No. 027 Exp. Date 3/5/2019

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>

Average 0.081

J. Tank pressure, Level 3 Func D Sub G P: 700 psi Reg 680 psi

K. Remarks/Maintenance record: ANNUAL INSPECTION

7/20/17
Date

Charles E. Ed
Field Inspector's Signature

24 July 17
Date

Robertta Huggins-Himmis
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:02

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 27
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 07/05/2017
Slope 705
Intercept -687158

IR Calibration Date: 07/05/2017

	3um	9um
0th Coef(*100):	-22113	-17955
1st Coef(*100):	271833	130091
2nd Coef(*100):	2129	1363
H2O adj(mg/l*10k):	691	568

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Sol.	0.010	14:12
03 Room Air	0.000	14:12
04 Std. Sol.	0.011	14:13
05 Room Air	0.000	14:13
06 Std. Sol.	0.010	14:14
07 Room Air	0.000	14:15

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 201603H
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.010 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:32
02 Std. Sol.	0.296	14:33
03 Room Air	0.000	14:34
04 Std. Sol.	0.295	14:34
05 Room Air	0.000	14:35
06 Std. Sol.	0.295	14:36
07 Room Air	0.000	14:36

08 Sim Temp = 34.0°C

Simul Ser No = MP3063
Std Sol No = 201603A
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
 0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:03

Test	AC	Time
01 Room Air	0.000	14:04
02 *Subject Test	INT*	14:05
03 Room Air	0.000	14:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 27
Expiration Date = 03/05/2019
County = 08 Oper No. = 666666

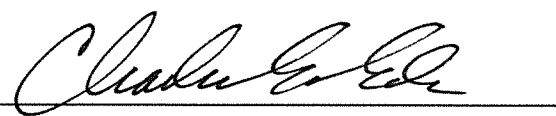

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:07

Test	AC	Time
01 Diagnostic	OK	14:08
02 Room Air	0.000	14:08
03 *Subject Test	RFI*	14:08
04 Room Air	0.000	14:09

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 27
Expiration Date = 03/05/2019
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Gas	0.081	14:42
03 Room Air	0.000	14:43
04 Std. Gas	0.081	14:43
05 Room Air	0.000	14:44
06 Std. Gas	0.081	14:44
07 Room Air	0.000	14:45

Lot No = 03417080A3
Cyl No = 27
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:46
02 Std. Gas	0.082	14:46
03 Room Air	0.000	14:46
04 Std. Gas	0.081	14:47
05 Room Air	0.000	14:47
06 Std. Gas	0.081	14:48
07 Room Air	0.000	14:48

Lot No = 03417080A3
Cyl No = 27
Exp Date = 03/05/2019
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
07/20/2017 14:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:49
02 Std. Gas	0.081	14:50
03 Room Air	0.000	14:50
04 Std. Gas	0.082	14:51
05 Room Air	0.000	14:51
06 Std. Gas	0.082	14:51
07 Room Air	0.000	14:52

Lot No = 03417080A3
Cyl No = 27
Exp Date = 03/05/2019
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC