



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) Peter Nielsen		
Location West Fargo PD		Intoxilyzer® Serial Number 80-006671
Gas Lot Number 20016080A2	Gas Cylinder Number 38	Gas Expiration Date 09052018

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.080 AC
 - B. 0.080 AC
 - C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 8-18-2017
Reviewed By (Crime Laboratory Use Only) 	Date 8-23-2017


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = WFPD 8164.14.00 09/16
08/18/2017 12:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:42
02 Std. Gas	0.080	12:42
03 Room Air	0.000	12:43
04 Std. Gas	0.080	12:43
05 Room Air	0.000	12:43
06 Std. Gas	0.081	12:44
07 Room Air	0.000	12:44

Lot No = 20016080A2
Cyl No = 38
Exp Date = 09/05/2018
County = 09 Oper No. = 039617



Operator Signature
PETER NIELSEN

Remarks:

Grieger-Nimmo, Roberta

From: Grieger-Nimmo, Roberta
Sent: Wednesday, August 16, 2017 2:05 PM
To: 'pete.nielsen@westfargond.gov'
Subject: Intox 8000 Gas Cylinder Install
Attachments: SFN59282EthanolBreathStanCylRpt.pdf; Dry Cal Check and 105G.pdf; SFN50496-IntoxRecord.pdf

Pete,

The most recent versions of our breath alcohol forms can be accessed on the Attorney General's website. They are grouped by Gas Cylinder Installation, Intox 8000 Installation, and additional materials. Here's a link for future reference: <https://attorneygeneral.nd.gov/alcoholtoxicology-testing/breath-alcohol-operator-forms>

Upon reviewing your gas cylinder paperwork, I noticed the barcode information from the new gas cylinder wasn't scanned into the instrument before running the ACA (dry calibration check). Please remove the gas cylinder and scan in the new lot information. You'll need to run the ACA test again. The old form 105-G (dated 1-2010) didn't have as clear of instruction on it as the new one (dated 10-2013). I attached the most recent version of the gas cylinder install (SFN59282) and also the intoxilyzer log book (SFN50496) as you will need to complete both forms. The form you sent me and the test record are attachment "Dry Cal Check and 105G" so you can see the difference between forms and that the lot number you wrote on the form doesn't match the lot number information at the bottom of the intox printout.

Feel free to call me if you have additional questions. I'm here today until 3PM and tomorrow from 6:30AM to 4:00PM.

Thank you!

Roberta Grieger-Nimmo
Forensic Scientist
Office of Attorney General
Crime Laboratory Division
2641 East Main
Bismarck, ND 58501
Phone (701) 328-6159
Fax (701) 328-6185



ETHANOL BREATH STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (1-2010)

Only Chemical Test Operators may change the Ethanol Breath Standard and complete this form.

Chemical Test Operator Name: <u>Pete Nielsen</u>		
Location: <u>West Fargo PD</u>		
Intoxilyzer® Serial Number: <u>80-006671</u>		
Gas Lot Number: <u>20016080A2</u>	Gas Cylinder Number: <u>38</u>	Expiration Date: <u>9-5-2018</u>
<input checked="" type="checkbox"/> New Cylinder Installed	<input type="checkbox"/> Cylinder Not Changed	
Instrument Test Results (Report to 3 Digits; Example, 0.081%)		
Test 1: 0. <u>080</u> %		
Test 2: 0. <u>081</u> %		
Test 3: 0. <u>080</u> %		
(ATTACH TEST RECORD)		


Chemical Test Operator Signature

8-12-17
Date

Reviewed By

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = WFPD 8164.14.00 09/16
08/12/2017 11:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:02
02 Std. Gas	0.080	11:03
03 Room Air	0.000	11:03
04 Std. Gas	0.081	11:04
05 Room Air	0.000	11:04
06 Std. Gas	0.080	11:05
07 Room Air	0.000	11:05

Lot No = 01316080A2
Cyl No = 35
Exp Date = 02/05/2018
County = 09 Oper No. = 039617



Operator Signature
PETER NIELSEN

Remarks: