



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Rebecca Kopp</i>		
Location <i>BMDC</i>	<i>Burleigh-Mornton County Detention Center location code BMCD 222</i>	Intoxilyzer® Serial Number <i>80-006674</i>
Gas Lot Number <i>20016080A2</i>	Gas Cylinder Number <i>#41</i>	Gas Expiration Date <i>9-5-18</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ACA  2. Perform an ACA Test (Level 1, Function C).  
*Test was performed*  
*Reso - Crime Lab*  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.081* AC  
 B. *0.081* AC  
 C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Rebecca Kopp</i>	Date <i>9-5-17</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Suenaga-Nemmo</i>	Date <i>9-20-17</i>

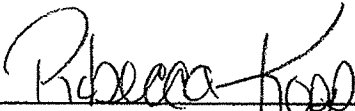
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
09/05/2017      12:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Gas	0.081	12:32
03 Room Air	0.000	12:32
04 Std. Gas	0.081	12:32
05 Room Air	0.000	12:33
06 Std. Gas	0.081	12:33
07 Room Air	0.000	12:34

Lot No = 20016080A2  
Cyl No = 41  
Exp Date = 09/05/2018  
County = 08      Oper No. = 041005

  
\_\_\_\_\_  
Operator Signature  
REBECCA KOPP

Remarks: