



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (10-2016)

Serial Number 80-006675	Instrument Location BSPD
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A.  Print Test (Level 1, Function P).
  - B.  ACA Test (Level 1, Function C).
  - C.  RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NA
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Pat Ray</i>	Date 9/14/2017
---	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Suegan-Timm</i>	Certified Date 15 Sep 17
---	-----------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006675  
Location = BSPD                    8164.14.00 09/16  
09/14/2017                            15:40

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

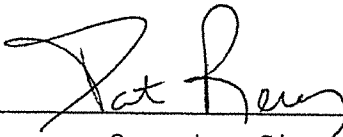
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            17316080A2  
Standard Cyl #?            25  
Standard Expiration?       08/05/2018  
Oper No?                    029710

Flow Cal. Date:            08/25/2016  
Slope                      701  
Intercept                  -701324

IR Calibration Date:       08/25/2016  
                                    3um                      9um

0th Coef(*100):	-8608	-21568
1st Coef(*100):	258710	128604
2nd Coef(*100):	2478	1279
H2O adj(mg/l*10k):	381	528

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
\_\_\_\_\_  
Operator Signature  
PATRICK RENZ

Remarks: *Install - Location Change*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006675  
Location = BSPD      8164.14.00 09/16  
09/14/2017      15:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Gas	0.076	15:41
03 Room Air	0.000	15:42
04 Std. Gas	0.077	15:42
05 Room Air	0.000	15:43
06 Std. Gas	0.077	15:43
07 Room Air	0.000	15:44

Lot No = 17316080A2  
Cyl No = 25  
Exp Date = 08/05/2018  
County = 08      Oper No. = 029710



Operator Signature  
PATRICK RENZ

Remarks: *Install - Location change*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006675  
Location = BSPD      8164.14.00 09/16  
09/14/2017      15:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:45
02 Std. Gas	RFI*	15:45
03 Room Air	0.000	15:46

\*Invalid Test  
Inhibited - RFI

Lot No = 17316080A2  
Cyl No = 25  
Exp Date = 08/05/2018  
County = 08      Oper No. = 029710



Operator Signature  
PATRICK RENZ

Remarks:

*Install - Location change*

Form 106-I8000