

INTOXILYZER® 8000 INSPECTION

Serial No.: 80-006668

Location: TOXL

- Check When Done
- A. Is the breath tube heated? X
  - B. Is the diagnostic check complete? X
  - C. Is the time, date, and year correct (re-set if necessary)? X
  - D. Print test. (Attach test record.) X
  - E. Low AC. Use  $\leq 0.03$  AC in ACA mode. (Attach test record.)  
Sim SN DR5188 Lot No. 16200 Rep AC 0.020AC X
  - F. High AC. Use  $\geq 0.25$  AC in ACA mode. (Attach test record.)  
Sim SN DR3378 Lot No. 201605F Rep AC 0.400AC X
  - G. Interferent check. Use 0.05% acetone in ethanol solution in ABA mode.  
(Attach test record.)  
Sim SN DR5144 Lot No. 1CS5 Solution 0.10AC + 0.05% Acetone X
  - H. RFI check. Run CMS mode. Key radio during CMS test.  
(Attach test record.) X
  - I. Calibration Check. Use Ethanol Breath Standard cylinder. Do three sets in ACA mode.  
(Attach test record.) 0.080 AC  
20016080A2 2016080A2 Cyl No. 48 Exp. Date 5 Sep 18  
Test 1 0.081 Test 4 0.082 Test 7 0.082  
Test 2 0.082 Test 5 0.081 Test 8 0.082  
Test 3 0.082 Test 6 0.081 Test 9 0.081  
Average 0.081
  - J. Tank pressure, Level 3 Func D Sub G P: 911 psi Reg 900 psi
  - K. Remarks/Maintenance record: Annual Inspection.

21 Sep 17  
Date

25 Sep. 2017  
Date

Roberta Hunger-Nimmo  
Field Inspector's Signature

Charles E. Eds  
Reviewed by

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-006668  
Location = TOXL               8164.14.00 09/16  
09/21/2017                       09:39

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?         No  
Display Volume?             No  
Disable On Memfull?         Yes  
# of Print Copies?          1  
Select Std (D/W/I)?         Dry  
Standard Value?             0.080  
Standard Lot #?             20016080A2  
Standard Cyl #?             48  
Standard Expiration?        09/05/2018  
Oper No?                    888888

Flow Cal. Date:             09/14/2017  
Slope                       694  
Intercept                  -693036

IR Calibration Date:        09/14/2017  
                              3um                    9um

	3um	9um
0th Coef(*100):	-18031	-20712
1st Coef(*100):	252469	127470
2nd Coef(*100):	3225	1600
H2O adj(mg/l*10k):	646	606

\*\*\*\*\* Printer Test End \*\*\*\*\*

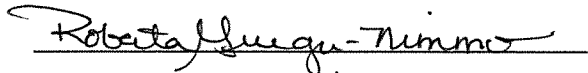  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Print test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006668  
Location = TOXL      8164.14.00 09/16  
09/21/2017      10:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:16
02 Std. Sol.	0.020	10:17
03 Room Air	0.000	10:18
04 Std. Sol.	0.020	10:18
05 Room Air	0.000	10:19
06 Std. Sol.	0.020	10:20
07 Room Air	0.000	10:20

08 Sim Temp = 34.0°C

Simul Ser No = DR5188  
Std Sol No = 16200  
County = 08      Oper No. = 888888

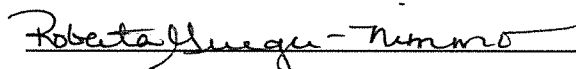

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:

LOW AC  
0.020 AC

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006668  
Location = TOXL                8164.14.00 09/16  
09/21/2017                        10:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:26
02 Std. Sol.	0.401	10:27
03 Room Air	0.000	10:27
04 Std. Sol.	0.403	10:28
05 Room Air	0.000	10:29
06 Std. Sol.	0.403	10:29
07 Room Air	0.000	10:30

08 Sim Temp = 34.0°C

Simul Ser No = DR3378  
Std Sol No = 201605F  
County = 08                      Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006668  
Location = TOXL                8164.14.00 09/16  
09/21/2017                        10:09

Test	AC	Time
01 Room Air	0.000	10:10
02 *Subject Test	INT*	10:10
03 Room Air	0.000	10:11

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                Weight = 150  
Test = DUI                      Cit = INTERFERENT CHK  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08                      Oper No. = 888888

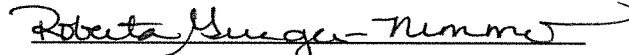

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check  
0.10AC + 0.0590 ACE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006668  
Location = TOXL                8164.14.00 09/16  
09/21/2017                        09:59

Test	AC	Time
01 Diagnostic	OK	10:00
02 Room Air	0.000	10:00
03 Subject Test 1	0.000	10:01
04 Room Air	0.000	10:03
05 Std. Gas	0.081	10:04
06 Room Air	0.000	10:05
07 *Subject Test	RFI*	10:06
08 Room Air	0.000	10:06

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

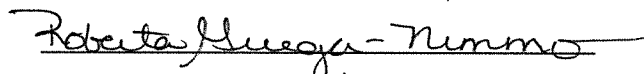

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006668  
Location = TOXL      8164.14.00 09/16  
09/21/2017      09:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:42
02 Std. Gas	0.081	09:42
03 Room Air	0.000	09:43
04 Std. Gas	0.082	09:43
05 Room Air	0.000	09:44
06 Std. Gas	0.082	09:44
07 Room Air	0.000	09:44

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

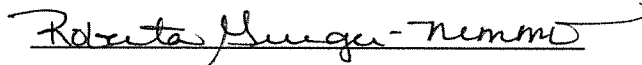

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006668  
Location = TOXL      8164.14.00 09/16  
09/21/2017      09:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:52
02 Std. Gas	0.082	09:52
03 Room Air	0.000	09:53
04 Std. Gas	0.081	09:53
05 Room Air	0.000	09:54
06 Std. Gas	0.081	09:54
07 Room Air	0.000	09:55

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

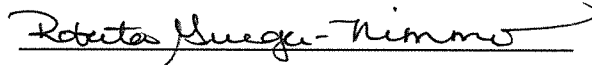

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006668  
Location = TOXL      8164.14.00 09/16  
09/21/2017      09:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:55
02 Std. Gas	0.082	09:56
03 Room Air	0.000	09:56
04 Std. Gas	0.081	09:57
05 Room Air	0.000	09:57
06 Std. Gas	0.081	09:57
07 Room Air	0.000	09:58

Lot No = 20016080A2  
Cyl No = 48  
Exp Date = 09/05/2018  
County = 08      Oper No. = 888888

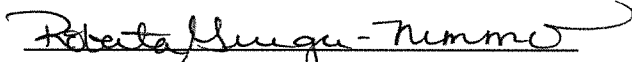  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC