



# ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) Kyle Christenson		
Location Mott, North Dakota	Intoxilyzer® Serial Number 80-006513	
Gas Lot Number 20016080A2	Gas Cylinder Number 43	Gas Expiration Date 09-05-2018

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0.081 AC
  - B. 0.081 AC
  - C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>K.C.</i> 4343	Date 09/28/2017
Reviewed By (Crime Laboratory Use Only) <i>Roberta Huggins Nimmo</i>	Date 10/3/2017


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = MOTT                 8164.13.00 06/09  
09/28/2017                                 13:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:19
02 Std. Gas	0.081	13:19
03 Room Air	0.000	13:20
04 Std. Gas	0.081	13:20
05 Room Air	0.000	13:21
06 Std. Gas	0.081	13:21
07 Room Air	0.000	13:22

Lot No = 20016080A2  
Cyl No = 43  
Exp Date = 09/05/2018  
County = 21                                 Oper No. = 130753

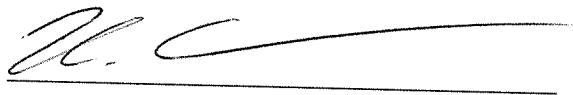

Operator Signature  
KYLE CHRISTENSON

Remarks: