



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION  
Toxicology Section/Breath Alcohol Program  
SFN 59281 (10-2016)

Serial Number <b>80-006691</b>	Instrument Location <i>mercer County Sheriffs Office</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>new Gas Standard also</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - Print Test (Level 1, Function P).
  - ACA Test (Level 1, Function C).
  - RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): Note: Gas cylinder installed
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

box checked  
in.  
by  
JBN  
3 Oct 17

*Ben, 3 Oct 17*

Field Inspector Signature <i>Camela B. Pehl</i>	Date <i>10/13/17</i>
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Debra Huggins-Himms</i>	Certified Date <i>10/13/17</i>
---	-----------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000        SN 80-006691  
Location = STAT                 8164.14.00 09/16  
10/13/2017                       14:02

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?


Current Instrument Setup

Data Entry Mode:                Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                 No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                 0.080  
Standard Lot #?                 03417080A3  
Standard Cyl #?                 007  
Standard Expiration?            03/05/2019  
Oper No?                         120906

Flow Cal. Date:                 09/07/2016  
Slope                            715  
Intercept                        -719337

IR Calibration Date:            09/07/2016  
                                  3um                    9um  
-----  
0th Coef(\*100):                -11169               -23079  
1st Coef(\*100):                265834               136373  
2nd Coef(\*100):                1816                 1210  
H2O adj (mg/l\*10k):            497                 550

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
\_\_\_\_\_  
Operator Signature  
PAMELA PEHL

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = STAT      8164.14.00 09/16  
10/13/2017      14:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Gas	0.083	14:05
03 Room Air	0.000	14:06
04 Std. Gas	0.083	14:06
05 Room Air	0.000	14:07
06 Std. Gas	0.083	14:07
07 Room Air	0.000	14:07


Lot No = 03417080A3

Cyl No = 007

Exp Date = 03/05/2019

County = 29

Oper No. = 120906

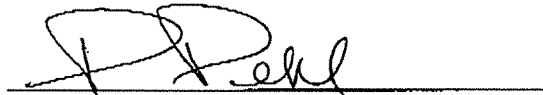

Operator Signature  
PAMELA PEHL

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = STAT      8164.14.00 09/16  
10/13/2017      14:16

Test	AC	Time
01 Room Air	RFI*	14:21
02 Room Air	RFI*	14:21

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST NA  
Sub DOB = 01/01/2017  
Sub Sex = Female      Weight = 150  
Test = OTH      Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 03417080A3  
Cyl No = 007  
Expiration Date = 03/05/2019  
County = 29      Oper No. = 120906

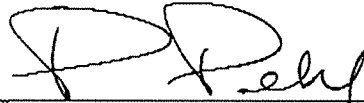

Operator Signature  
PAMELA PEHL

Remarks:



# Mercer County Sheriff's Department

Sheriff - Dean Danzeisen Chief of Operations - Colin Peterson Chief Deputy - Terrance Ternes

Phone 701-745-3333 - Fax 701-745-3347 - Email Address: mercer@nd.gov  
PO Box 39 - Stanton, ND 58571-0039

\*\*\*\*\*

## FAX TRANSMITTAL MEMO

TOTAL NUMBER OF PAGES INCLUDING THIS COVER SHEET: 5

ATTN: Robertu

DATE: 10/13/17

FAX #: 701-328-6185

FROM: Pam Pehe / Mercer City

MESSAGE: Moved into a new gas standard