

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Nu	anibei	80-06684	Instrument Location West Funge Po					
Reason for Install/Repair								
	Install After Receiving From Crime Laboratory Install After Location Change							
U Ot	Other (Specify)							
Check V	Check When Done:							
Ø	1.	Surge Protector Installed/Properly Groun	ded.					
X	2.	Telephone Line Connected to Intoxilyzer	® 8000.					
K	3.	. Breath Tube Heated.						
內	4.	Enter Preliminary Data (i.e. Date, Time, [OST (Y), and Location; Level 2,	Function E).				
内	5.	5. Scan/Enter Gas Cylinder Information (Level 1, Function S).						
	6.	Run Tests:						
Tests were a	done.	A. Print Test (Level 1, Function P).						
and RPT we	re	B. ACA Test (Level 1, Function C). C. RFI Test (CMS Mode or Level 1, Fun	otion Box C. Koy Bodio Dyning	To a4)				
received 1/	7/18/1	Repair and/or Maintenance Performed (if	. l à .	rest).				
Ø		omplete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the toxilyzer® for Use.						
团	9. F	File Previous Intoxilyzer® Record (SFN5)	e Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.					
ĺγ.	10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.							
Field Insp	ector S	Signature		Date 12-30-17				
<u> </u>	70.5017							
Crime Lab	Crime Laboratory Use Only							
This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of								
Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.								
Reviewed	l/Certifi	ed by Roberta Hugu nin	nino-	Certified Date				

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
12/30/2017 10:27

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes
of Print Copies? 1 # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 19817080A4 Standard Cyl #? Standard Expiration? 09/05/2019 039617 Oper No?

Flow Cal. Date: 09/06/2016 Slope 709 Intercept -725441

IR Calibration Date: 09/06/2016 3um 9um

Oth Coef(*100): -16013 -23467 1st Coef(*100): 256431 135731 2nd Coef(*100): 2904 1268 H2O adj(mg/l*10k): 553 514

****** Printer Test End *******

Operator Signature PETER NIELSEN

Remarks:

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = WFPD
12/30/2017

Alcohol Analyzer SN 80-006684 8164.14.00 09/16 10:27

DRY CAL CHECK

Test		AC	Time
01 Room 02 Std. 03 Room 04 Std. 05 Room 06 Std.	Gas Air Gas Air	0.000 0.082 0.000 0.082 0.000 0.082	10:28 10:28 10:29 10:29 10:30
07 Room		0.000	10:31

Lot No = 19817080A4

Cyl No = 4

Exp Date = 09/05/2019

County = 09

Oper No. = 039617

Operator Signature PETER NIELSEN

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-006684
Location = WFPD	8164.14.00 09/16
12/30/2017	10:32

Test	AC	Time
01 Room Air	0.000	10:33
02 *Subject Test	RFI*	10:34
03 Room Air	0.000	10:34

*Invalid Test Inhibited - RFI

Sub Name = TEST, TEST NA .

Sub DOB = 01/01/2001

Sub Sex = Male Weight = 200
Test = DUI Cit = TEST

Dr. Lic. = ND/TEST

Lot No = 19817080A4

Cyl No = 4

Expiration Date = 09/05/2019

Operator Signature PETER NIELSEN

Remarks:

Form 106-I8000