



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (10-2016)

Serial Number	Instrument Location
80-006511	Robette County
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <i>MOVED to new JAIL</i>	
<input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Glenn</i>	Date <i>1-16-18</i>
--	---------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Muega-Nimmo</i>	Certified Date <i>1-19-18</i>
--	-------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = ROLL 8164.14.00 09/16
01/16/2018 08:58

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 12
Standard Expiration? 03/05/2019
Oper No? 069108

Flow Cal. Date: 08/21/2015
Slope 688
Intercept -714727

IR Calibration Date: 08/21/2015
 3um 9um

	3um	9um
0th Coef(*100):	-19744	-15946
1st Coef(*100):	272125	142950
2nd Coef(*100):	2975	1383
H2O adj(mg/l*10k):	725	394

***** Printer Test End *****

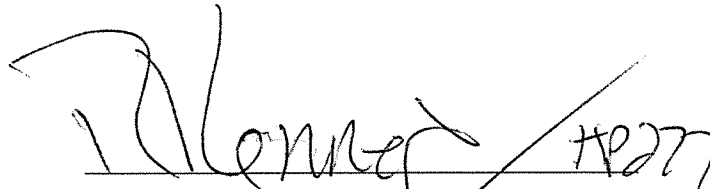

Operator Signature
ROBERT KENNEDY

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = ROLL 8164.14.00 09/16
01/16/2018 08:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:59
02 Std. Gas	0.083	08:59
03 Room Air	0.000	09:00
04 Std. Gas	0.082	09:00
05 Room Air	0.000	09:01
06 Std. Gas	0.083	09:01
07 Room Air	0.000	09:02

Lot No = 03417080A3
Cyl No = 12
Exp Date = 03/05/2019
County = 40 Oper No. = 069108



Operator Signature
ROBERT KENNEDY

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = ROLL 8164.14.00 09/16
01/16/2018 09:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	09:05
02 Room Air	0.000	09:06

*Invalid Test
Inhibited - RFI

Lot No = 03417080A3
Cyl No = 12
Exp Date = 03/05/2019
County = 48 Oper No. = 069108

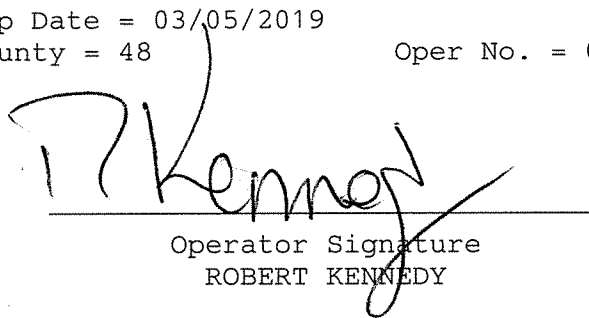

Operator Signature
ROBERT KENNEDY

Remarks: