



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <u>Travis Skar</u>		
Location <u>Ward Co Jail</u>	Intoxilyzer® Serial Number <u>80-006677</u>	
Gas Lot Number <u>19817080A4</u>	Gas Cylinder Number <u>23</u>	Gas Expiration Date <u>9/5/2019</u>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. 0.081 AC

B. 0.081 AC

C. 0.081 AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <u>[Signature]</u>	Date <u>01/24/2018</u>
Reviewed By (Crime Laboratory Use Only) <u>[Signature]</u>	Date <u>01/25/2018</u>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006677  
Location = WARD      8164.14.00 09/16  
01/24/2018      18:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:36
02 Std. Gas	0.081	18:36
03 Room Air	0.000	18:37
04 Std. Gas	0.081	18:37
05 Room Air	0.000	18:38
06 Std. Gas	0.081	18:38
07 Room Air	0.000	18:39

Lot No = 19817080A4  
Cyl No = 23  
Exp Date = 09/05/2019  
County = 51      Oper No. = 120513

  
\_\_\_\_\_  
Operator Signature  
TRAVIS SKAR

Remarks: