



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (10-2016)

Serial Number <b>80-006667</b>	Instrument Location <b>Finley</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Install after bringing to state lab for download</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NA
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>1/25/18</b>
-------------------------------	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by 	Certified Date <b>1/26/18</b>
---------------------------	----------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006667  
Location = FINL      8164.14.00 09/16  
01/25/2018      11:39

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      09216080A3  
Standard Cyl #?      41  
Standard Expiration?      05/05/2018  
Oper No?      061123

Flow Cal. Date:      08/15/2016  
Slope      699  
Intercept      -645669

IR Calibration Date:      08/15/2016  
  3um    9um

-----  
0th Coef(\*100):      -23625                          -25093  
1st Coef(\*100):      269805                          129386  
2nd Coef(\*100):      3177                                  1385  
H2O adj(mg/l\*10k):      621                                  546

\*\*\*\*\* Printer Test End \*\*\*\*\*

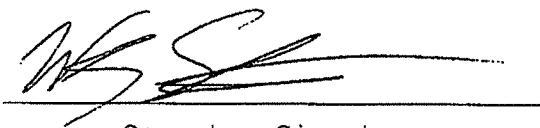

Operator Signature  
WESLEY STRAIGHT

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006667  
Location = FINL                    8164.14.00 09/16  
01/25/2018                            11:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:50
02 Std. Gas	0.081	11:50
03 Room Air	0.000	11:51
04 Std. Gas	0.081	11:51
05 Room Air	0.000	11:52
06 Std. Gas	0.081	11:52
07 Room Air	0.000	11:52

Lot No = 09216080A3  
Cyl No = 41  
Exp Date = 05/05/2018  
County = 46                            Oper No. = 061123

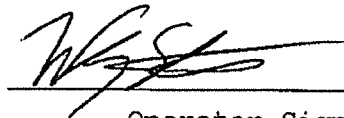  
\_\_\_\_\_  
Operator Signature  
WESLEY STRAIGHT

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006667  
Location = FINL      8164.14.00 09/16  
01/25/2018      11:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:53
02 Room Air	0.000	11:54

\*Invalid Test  
Inhibited - RFI

Lot No = 09216080A3  
Cyl No = 41  
Exp Date = 05/05/2018  
County = 46      Oper No. = 061123

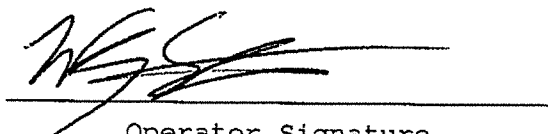

Operator Signature  
WESLEY STRAIGHT

Remarks: