



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (10-2016)

Serial Number <i>40-006685</i>	Instrument Location <i>NAPD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- nr* 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

*Tests were ran 15 Feb 18 RWB*

Field Inspector Signature <i>[Signature]</i>	Date <i>2-15-18</i>
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Bueque-Nimmo</i>	Certified Date <i>2-15-18</i>
--	----------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006685  
Location = NAPO                    8164.14.00 09/16  
02/15/2018                            12:51

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=\_|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?


Current Instrument Setup

Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    09216080A3  
Standard Cyl #?                    46  
Standard Expiration?               05/05/2018  
Oper No?                            131282  
  
Flow Cal. Date:                    09/06/2016  
Slope                                705  
Intercept                          -687370

IR Calibration Date:               09/06/2016  
    3um                                9um

-----  
0th Coef(\*100):            -14155                                -22915  
1st Coef(\*100):            271436                                135296  
2nd Coef(\*100):            2016                                    1251  
H2O adj(mg/l\*10k):       372                                    505

\*\*\*\*\* Printer Test End \*\*\*\*\*

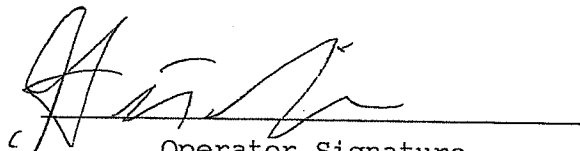  
\_\_\_\_\_  
Operator Signature  
HUNTER LEWIS

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006685  
Location = NAPO      8164.14.00 09/16  
02/15/2018      12:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:52
02 Std. Gas	0.079	12:52
03 Room Air	0.000	12:53
04 Std. Gas	0.080	12:53
05 Room Air	0.000	12:54
06 Std. Gas	0.081	12:55
07 Room Air	0.000	12:55

Lot No = 09216080A3  
Cyl No = 46  
Exp Date = 05/05/2018  
County = 24      Oper No. = 131282

  
\_\_\_\_\_  
Operator Signature  
HUNTER LEWIS

Remarks:


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006685  
Location = NAPO      8164.14.00 09/16  
02/15/2018      12:57

Test	AC	Time
01 Room Air	0.000	12:58
02 *Subject Test	RFI*	12:58
03 Room Air	0.000	12:59

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/1980  
Sub Sex = Male      Weight = 250  
Test = DUI      Cit = TEST  
Dr. Lic. = ND/TEST  
Lot No = 09216080A3  
Cyl No = 46  
Expiration Date = 05/05/2018  
County = 24      Oper No. = 131282

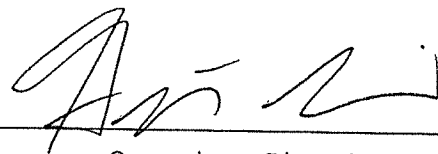

Operator Signature  
HUNTER LEWIS

Remarks: