



ETHANOL GAS STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (10-2013)

| | | |
|---|-----------------------------------|--|
| Chemical Test Operator Name (Print) <i>Dustin B. Newman</i> | | |
| Location <i>1201 12th St SE Watford City</i> | | Intoxilyzer® Serial Number <i>80-006507</i> |
| Gas Lot Number <i>19817080A4</i> | Gas Cylinder Number <i>018</i> | Gas Expiration Date <i>09/05/2019</i> |

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.081* AC

B. *0.081* AC

C. *0.080* AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

| | |
|--|---------------------------|
| Chemical Test Operator Signature <i>Dustin B. Newman</i> | Date <i>03/07/2018</i> |
| Reviewed By (Crime Laboratory Use Only) <i>Robert H. Meyer - Tamm</i> | Date <i>03/08/2018</i> |


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = WATF 8164.13.00 06/09
03/07/2018 08:29

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 08:30 |
| 02 Std. Gas | 0.081 | 08:30 |
| 03 Room Air | 0.000 | 08:31 |
| 04 Std. Gas | 0.081 | 08:31 |
| 05 Room Air | 0.000 | 08:31 |
| 06 Std. Gas | 0.080 | 08:32 |
| 07 Room Air | 0.000 | 08:32 |

Lot No = 19817080A4
Cyl No = 18
Exp Date = 09/05/2019
County = 27 Oper No. = 030924

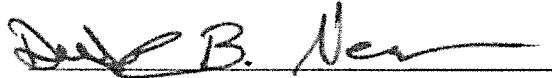

Operator Signature
DUSTIN NEWMAN

Remarks: *Good ACA Test*