



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (10-2016)

Serial Number <b>00-006681</b>	Instrument Location <b>FINL</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- ~~M2~~ 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): NA
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>4/9/18</b>
-------------------------------	-----------------------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by 	Certified Date <b>4/10/18</b>
---------------------------	----------------------------------




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = FINL      8164.14.00 09/16  
04/09/2018      13:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:34
02 Std. Gas	0.081	13:34
03 Room Air	0.000	13:34
04 Std. Gas	0.080	13:35
05 Room Air	0.000	13:35
06 Std. Gas	0.081	13:36
07 Room Air	0.000	13:36

Lot No = 34917080A3  
Cyl No = 1  
Exp Date = 02/05/2020  
County = 46      Oper No. = 061123

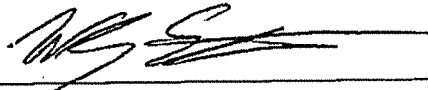

Operator Signature  
WESLEY STRAIGHT

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = FINL      8164.14.00 09/16  
04/09/2018      . 13:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	13:32
02 Room Air	0.000	13:33

\*Invalid Test  
Inhibited - RFI

Lot No = 34917080A3  
Cyl No = 1  
Exp Date = 02/05/2020  
County = 46      Oper No. = 061123



Operator Signature  
WESLEY STRAIGHT

Remarks: