



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number <i>80-006667</i>	Instrument Location <i>Garden Valley County Sheriff's Office</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NONE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>04-11-2018</i>
Reviewed By (Crime Laboratory Use Only) <i>Robert [Signature]</i>	Date <i>04-20-2018</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/11/2018 17:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:27
02 Std. Gas	0.079	17:28
03 Room Air	0.000	17:28
04 Std. Gas	0.079	17:29
05 Room Air	0.000	17:29
06 Std. Gas	0.079	17:29
07 Room Air	0.000	17:30

Lot No = 34917080A3
Cyl No = 8
Exp Date = 02/05/2020
County = 17 Oper No. = 130349

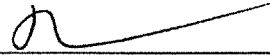

Operator Signature
DEY MUCKLE

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/11/2018 17:31

Test	AC	Time
01 Diagnostic	OK	17:34
02 Room Air	0.000	17:34
03 *Subject Test	0.000*	17:37
04 Room Air	0.000	17:40
05 Std. Gas	0.078	17:41
06 Room Air	0.000	17:42
07 *Subject Test	RFI**	17:43
08 Room Air	0.000	17:43

*Deficient Sample - Value Printed was
Highest Obtained

**Invalid Test
Inhibited - RFI

Sub Name = TEST, INTOX INSTALL NA
Sub DOB = 01/01/1901
Sub Sex = Male Weight = 225
Test = DUI Cit = 9999999
Dr. Lic. = ND/INSTALL RFI TEST
Lot No = 34917080A3
Cyl No = 8
Expiration Date = 02/05/2020
County = 17 Oper No. = 130349

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
DEY MUCKLE

Remarks: