



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number <i>80-006673</i>	Instrument Location <i>Burke Co. Sheriff's Office</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

*ONA
BY
JAN*

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jeremy Grohs</i>	Date <i>4/17/18</i>
--	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Muegge-Nimmer</i>	Certified Date <i>4/24/18</i>
---	----------------------------------

NOTE: Additional RFI tests were run by Jeremy Grohs 4/20/18.
BSW 24 Apr 18

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
04/17/2018 11:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-==|
ABCDEFGHIJKLMNPNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

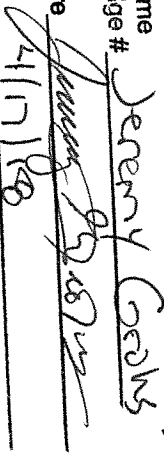
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 36
Standard Expiration? 08/05/2018
Oper No? 130230

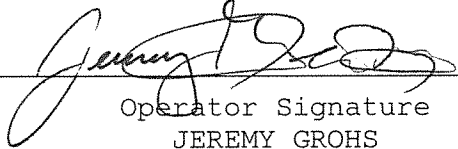
Flow Cal. Date: 03/30/2017
Slope 697
Intercept -667725

IR Calibration Date: 03/30/2017
3um 9um

0th Coef(*100): -9136 -17613
1st Coef(*100): 267126 133333
2nd Coef(*100): 5438 1698
H2O adj(mg/l*10k): 684 589

***** Printer Test End *****

I certify that the attached is a true and correct copy of the Intoxilyzer test record.
Print Name and Badge # Jeremy Grohs #4170
Signature 
Date 4/17/18

 #4170
Operator Signature
JEREMY GROHS

Remarks:

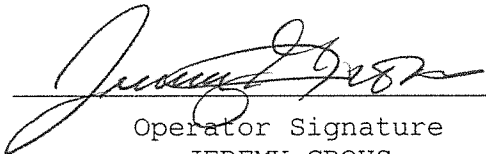
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
04/17/2018 12:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:05
02 Std. Gas	0.082	12:05
03 Room Air	0.000	12:05
04 Std. Gas	0.082	12:06
05 Room Air	0.000	12:06
06 Std. Gas	0.082	12:07
07 Room Air	0.000	12:07


Lot No = 17316080A2
Cyl No = 36
Exp Date = 08/05/2018
County = 07 Oper No. = 130230


Operator Signature
JEREMY GROHS

Remarks: RFI TEST
GOOD TEST

Form 106-I8000

I certify that the attached is a true and correct copy of the Intoxilyzer test record.

Print Name and Badge # Jeremy Grohs
Signature 
Date 4/17/18

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
04/17/2018 11:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:00
02 Std. Gas	0.082	12:00
03 Room Air	0.000	12:01
04 Std. Gas	0.081	12:01
05 Room Air	0.000	12:02
06 Std. Gas	0.081	12:02
07 Room Air	0.000	12:03

Lot No = 17316080A2
Cyl No = 36
Exp Date = 08/05/2018
County = 07 Oper No. = 130230


Operator Signature
JEREMY GROHS

Remarks: *Good Test*

I certify that the attached is a true and correct copy of the Intoxilyzer test record.

Print Name and Badge # *Jeremy Grohs #4170*
Signature *[Signature]*
Date *4/17/18*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
04/20/2018 18:39

Test	AC	Time
01 Room Air	0.000	18:40
02 Reference	RFI*	18:40
03 Room Air	0.000	18:41

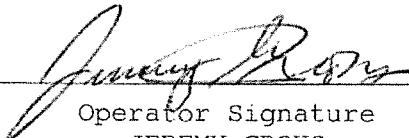
*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 09/05/1980 Weight = 200
Sub Sex = Male Cit = 9999999
Test = OTH
Dr. Lic. = ND/GRO805866
Lot No = 17316080A2
Cyl No = 36
Expiration Date = 08/05/2018
County = 07 Oper No. = 130230

Print Name
and Badge #
Signature
Date

Jeremy Grohs #4170
4/20/18

I certify that the attached is a true and correct copy of the Intoxilyzer test record.


Operator Signature
JEREMY GROHS

Remarks: Good Test
RFI Test
Set up of Intox 8000
Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

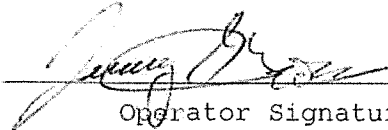
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
04/20/2018 18:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	18:41
02 Room Air	0.000	18:42

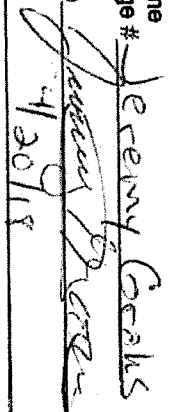
*Invalid Test
Inhibited - RFI

Lot No = 17316080A2
Cyl No = 36
Exp Date = 08/05/2018
County = 07 Oper No. = 130230


Operator Signature
JEREMY GROHS

Remarks: **Good Test**
Set up of Intox 8000

Form 106-18000

Print Name and Badge # Jeremy Grohs #4120
Signature 
Date 4/20/18

I certify that the attached is a true and correct copy of the Intoxilyzer test record.