



ETHANOL GAS STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Hunker Lewis</i>		
Location <i>WHPD</i>	Intoxilyzer® Serial Number <i>00-006685</i>	
Gas Lot Number <i>19911060A4</i>	Gas Cylinder Number <i>012</i>	Gas Expiration Date <i>4/5/20A</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.090* AC
 - B. *0.081* AC
 - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>4-24-18</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>4-24-18</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
04/24/2018 10:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:04
02 Std. Gas	0.080	10:04
03 Room Air	0.000	10:05
04 Std. Gas	0.081	10:05
05 Room Air	0.000	10:06
06 Std. Gas	0.081	10:06
07 Room Air	0.000	10:07

Lot No = 19817080A4
Cyl No = 12
Exp Date = 09/05/2019
County = 24 Oper No. = 131282

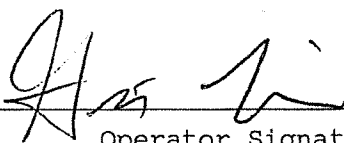

Operator Signature
HUNTER LEWIS

Remarks: