

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006685

Location: NAPO
LOGAN COUNTY
S. O.

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1005 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10.4.2018
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR3191 Lot #: 17350 Exp. Date: 10.11.2019

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC, + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS#5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 1981708044 Cyl #: 12 Exp. Date: 9/5/2019

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.080</u>	Test 9	<u>0.081</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Eds
Breath Analyst Signature

5.3.18
Date

Roberto Yungu-Tummo
Reviewed by

5.9.18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:04

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 12
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 705
Intercept -687370

IR Calibration Date: 09/06/2016
 3um 9um

0th Coef(*100): -14155 -22915
1st Coef(*100): 271436 135296
2nd Coef(*100): 2016 1251
H2O adj(mg/l*10k): 372 505

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Sol.	0.017	12:31
03 Room Air	0.000	12:32
04 Std. Sol.	0.018	12:33
05 Room Air	0.000	12:33
06 Std. Sol.	0.018	12:34
07 Room Air	0.000	12:35


08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 201610A

County = 24

Oper No. = 666666

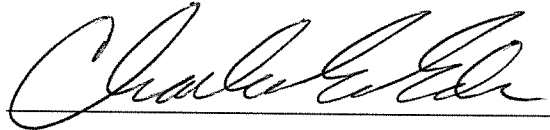

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:37
02 Std. Sol.	0.295	12:38
03 Room Air	0.000	12:39
04 Std. Sol.	0.296	12:40
05 Room Air	0.000	12:40
06 Std. Sol.	0.297	12:41
07 Room Air	0.000	12:42

08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 24

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:12

Test	AC	Time
01 Room Air	0.000	12:12
02 *Subject Test	INT*	12:13
03 Room Air	0.000	12:14

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 12
Expiration Date = 09/05/2019
County = 24 Oper No. = 666666

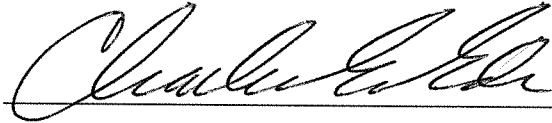

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:08

Test	AC	Time
01 Diagnostic	OK	12:09
02 Room Air	0.000	12:10
03 *Subject Test	RFI*	12:10
04 Room Air	0.000	12:11

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 12
Expiration Date = 09/05/2019
County = 24 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:20
02 Std. Gas	0.081	12:21
03 Room Air	0.000	12:21
04 Std. Gas	0.081	12:22
05 Room Air	0.000	12:22
06 Std. Gas	0.081	12:23
07 Room Air	0.000	12:23

Lot No = 19817080A4

Cyl No = 12

Exp Date = 09/05/2019

County = 24

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:24
02 Std. Gas	0.081	12:25
03 Room Air	0.000	12:25
04 Std. Gas	0.081	12:26
05 Room Air	0.000	12:26
06 Std. Gas	0.080	12:27
07 Room Air	0.000	12:27

Lot No = 19817080A4
Cyl No = 12
Exp Date = 09/05/2019
County = 24 Oper No. = 666666

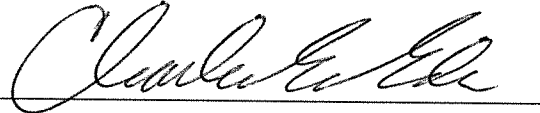

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006685
Location = NAPO 8164.14.00 09/16
05/03/2018 12:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Gas	0.081	12:44
03 Room Air	0.000	12:45
04 Std. Gas	0.081	12:45
05 Room Air	0.000	12:46
06 Std. Gas	0.081	12:46
07 Room Air	0.000	12:47

Lot No = 19817080A4

Cyl No = 12

Exp Date = 09/05/2019

County = 24

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000