

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006670

Location: GARR

GARRISON - McLEAN
COUNTY
S.O.

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1001 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10-4-18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10-11-19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS#5 Exp. Date: 5-15-19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 11 Exp. Date: 2/5/20

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Edle
Breath Analyst Signature

5-16-2018
Date

Roberta Suep-Thomas
Reviewed by

24 May 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34917080A3
Standard Cyl #? 11
Standard Expiration? 02/05/2020
Oper No? 666666

Flow Cal. Date: 08/15/2016
Slope 691
Intercept -681667

IR Calibration Date: 08/15/2016
 3um 9um

0th Coef(*100): -36090 -26909
1st Coef(*100): 270161 131597
2nd Coef(*100): 3492 1526
H2O adj(mg/l*10k): 790 617

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:34
02 Std. Sol.	0.018	10:34
03 Room Air	0.000	10:35
04 Std. Sol.	0.018	10:36
05 Room Air	0.000	10:36
06 Std. Sol.	0.018	10:37
07 Room Air	0.000	10:37

08 Sim Temp = 34.0°C

Simul Ser No = DR3377
Std Sol No = 201610A
County = 28 Oper No. = 666666

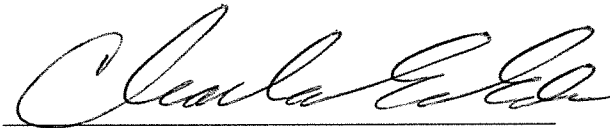

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Sol.	0.291	10:49
03 Room Air	0.000	10:50
04 Std. Sol.	0.292	10:51
05 Room Air	0.000	10:51
06 Std. Sol.	0.292	10:52
07 Room Air	0.000	10:52


08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 28

Oper No. = 666666

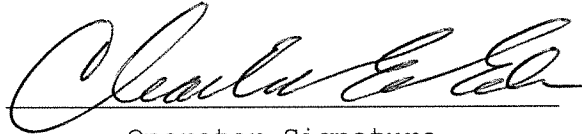

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:23

Test	AC	Time
01 Room Air	0.000	10:24
02 *Subject Test	INT*	10:24
03 Room Air	0.000	10:25

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 11
Expiration Date = 02/05/2020
County = 28 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:20

Test	AC	Time
01 Diagnostic	OK	10:21
02 Room Air	0.000	10:22
03 *Subject Test	RFI*	10:22
04 Room Air	0.000	10:22

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 11
Expiration Date = 02/05/2020
County = 28 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

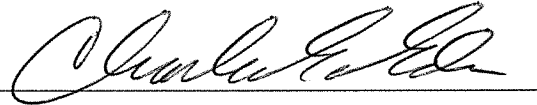

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:25
02 Std. Gas	0.080	10:26
03 Room Air	0.000	10:26
04 Std. Gas	0.079	10:27
05 Room Air	0.000	10:27
06 Std. Gas	0.080	10:28
07 Room Air	0.000	10:28

Lot No = 34917080A3
Cyl No = 11
Exp Date = 02/05/2020
County = 28 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:29
02 Std. Gas	0.080	10:29
03 Room Air	0.000	10:30
04 Std. Gas	0.080	10:30
05 Room Air	0.000	10:31
06 Std. Gas	0.080	10:31
07 Room Air	0.000	10:32

Lot No = 34917080A3
Cyl No = 11
Exp Date = 02/05/2020
County = 28 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006670
Location = GARR 8164.14.00 09/16
05/16/2018 10:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Gas	0.080	10:56
03 Room Air	0.000	10:56
04 Std. Gas	0.080	10:56
05 Room Air	0.000	10:57
06 Std. Gas	0.080	10:57
07 Room Air	0.000	10:58


Lot No = 34917080A3

Cyl No = 11

Exp Date = 02/05/2020

County = 28

Oper No. = 666666

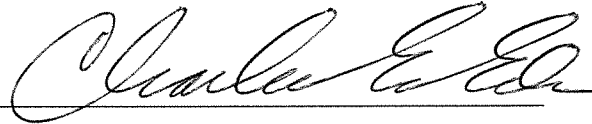

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000