

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-0066680

Location: Lansure CO SO  
LAMO

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: ~~1000~~ <sup>999</sup> psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: 1055 Exp. Date: 15 May 19

Display reads "Interferent Detect"

3.  RFI Check (CMS Mode)  Display reads "RFI Detect"

4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 19817080A4 Cyl #: 7 Exp. Date: 5 Sep 19

Test 1 0.080 Test 4 0.079 Test 7 0.078

Test 2 0.079 Test 5 0.078 Test 8 0.079

Test 3 0.078 Test 6 0.078 Test 9 0.079

Average 0.079

C. Remarks/Maintenance: Annual inspection. Replaced o-ring  
On breath hose.

Instrument is acceptable to be used in the field.

Roberta Guzman-Romero  
Breath Analyst Signature

6 Jun 18  
Date

Charles E. Ed  
Reviewed by

08 JUN 18  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006680  
Location = LAMO                8164.14.00 09/16  
06/06/2018                      12:43

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19817080A4  
Standard Cyl #?            7  
Standard Expiration?       09/05/2019  
Oper No?                    888888

Flow Cal. Date:            09/06/2016  
Slope                      681  
Intercept                  -620591

IR Calibration Date:       09/06/2016  
                                    3um                      9um

-----  
0th Coef(\*100):            -14265                      -21271  
1st Coef(\*100):            260743                      129687  
2nd Coef(\*100):            3906                        1392  
H2O adj(mg/l\*10k):        742                         684

\*\*\*\*\* Printer Test End \*\*\*\*\*

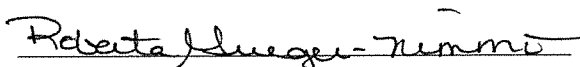

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks:                  Print Test

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = LAMO      8164.14.00 09/16  
06/06/2018      13:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:05
02 Std. Sol.	0.019	13:05
03 Room Air	0.000	13:06
04 Std. Sol.	0.018	13:07
05 Room Air	0.000	13:07
06 Std. Sol.	0.018	13:08
07 Room Air	0.000	13:08


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201610A

County = 23

Oper No. = 888888

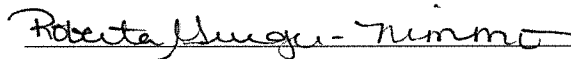

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = LAMO      8164.14.00 09/16  
06/06/2018      12:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:59
02 Std. Sol.	0.294	13:00
03 Room Air	0.000	13:01
04 Std. Sol.	0.294	13:01
05 Room Air	0.000	13:02
06 Std. Sol.	0.294	13:03
07 Room Air	0.000	13:03


08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 23

Oper No. = 888888

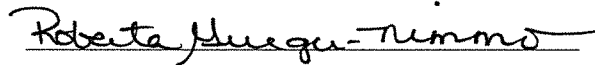

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.340 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006680  
Location = LAMO                8164.14.00 09/16  
06/06/2018                        12:53

Test	AC	Time
01 Room Air	0.000	12:54
02 *Subject Test	INT*	12:55
03 Room Air	0.000	12:55

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 7  
Expiration Date = 09/05/2019  
County = 23                            Oper No. = 888888

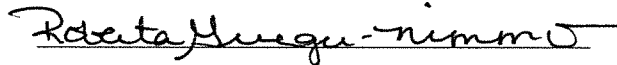

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*  
*0.10 AC + 0.05% Acetone*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006680  
Location = LAMO                    8164.14.00 09/16  
06/06/2018                            12:56

Test	AC	Time
01 Diagnostic	OK	12:56
02 Room Air	0.000	12:57
03 *Subject Test	RFI*	12:57
04 Room Air	0.000	12:58

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 7  
Expiration Date = 09/05/2019  
County = 23                            Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

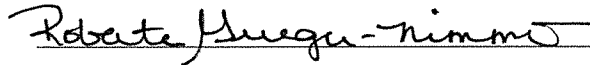

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *RFI Check*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006680  
Location = LAMO                8164.14.00 09/16  
06/06/2018                        12:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Gas	0.080	12:44
03 Room Air	0.000	12:45
04 Std. Gas	0.079	12:45
05 Room Air	0.000	12:45
06 Std. Gas	0.078	12:46
07 Room Air	0.000	12:46

Lot No = 19817080A4  
Cyl No = 7  
Exp Date = 09/05/2019  
County = 23                            Oper No. = 888888

Roberta Grieger-Nimmo  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080AC


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = LAMO      8164.14.00 09/16  
06/06/2018      12:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:47
02 Std. Gas	0.079	12:48
03 Room Air	0.000	12:48
04 Std. Gas	0.078	12:48
05 Room Air	0.000	12:49
06 Std. Gas	0.078	12:49
07 Room Air	0.000	12:50

Lot No = 19817080A4  
Cyl No = 7  
Exp Date = 09/05/2019  
County = 23      Oper No. = 888888

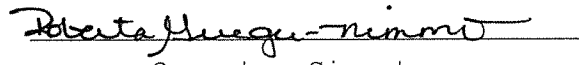  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC  
Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006680  
Location = LAMO      8164.14.00 09/16  
06/06/2018      12:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:51
02 Std. Gas	0.078	12:51
03 Room Air	0.000	12:51
04 Std. Gas	0.079	12:52
05 Room Air	0.000	12:52
06 Std. Gas	0.079	12:53
07 Room Air	0.000	12:53

Lot No = 19817080A4  
Cyl No = 7  
Exp Date = 09/05/2019  
County = 23      Oper No. = 888888

Roberta Grieger-Nimmo  
Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000