

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006508

Richland Co LEC
Location: WAMP

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 989 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1CSS Exp. Date: 15 May 19

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 19817080A4 Cyl #: 24 Exp. Date: 5 Sep 19

Test 1 0.080 Test 4 0.080 Test 7 0.080

Test 2 0.079 Test 5 0.079 Test 8 0.080

Test 3 0.080 Test 6 0.079 Test 9 0.080

Average 0.080

C. Remarks/Maintenance: Annual inspection. Replaced o-ring on breath hose.

Instrument is acceptable to be used in the field.

Roberta Muegu-timoro
Breath Analyst Signature

12 Jun 18
Date

Charles E. Edr
Reviewed by

12 Jun 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

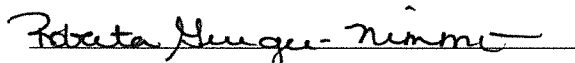
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 24
Standard Expiration? 09/05/2019
Oper No? 888888

Flow Cal. Date: 08/20/2015
Slope 644
Intercept -568181

IR Calibration Date: 08/20/2015

	3um	9um
0th Coef(*100):	-22365	-18627
1st Coef(*100):	256969	136095
2nd Coef(*100):	3168	1444
H2O adj(mg/l*10k):	693	453

***** Printer Test End *****



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:36

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:36
02 Std. Sol.	0.019	08:37
03 Room Air	0.000	08:38
04 Std. Sol.	0.019	08:38
05 Room Air	0.000	08:39
06 Std. Sol.	0.019	08:40
07 Room Air	0.000	08:40

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201610A

County = 39

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

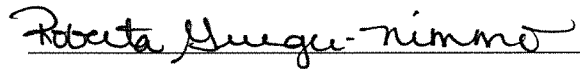
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:29
02 Std. Sol.	0.302	08:30
03 Room Air	0.000	08:31
04 Std. Sol.	0.302	08:32
05 Room Air	0.000	08:32
06 Std. Sol.	0.301	08:33
07 Room Air	0.000	08:34

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 39 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

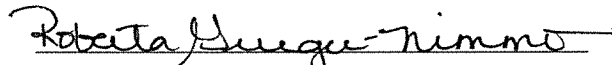
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:17

Test	AC	Time
01 Room Air	0.000	08:18
02 *Subject Test	INT*	08:19
03 Room Air	0.000	08:20

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 24
Expiration Date = 09/05/2019
County = 39 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

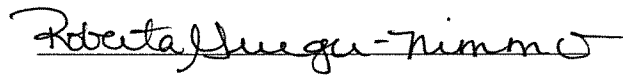
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:15

Test	AC	Time
01 Diagnostic	OK	08:16
02 Room Air	0.000	08:16
03 *Subject Test	RFI*	08:17
04 Room Air	0.000	08:17

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 24
Expiration Date = 09/05/2019
County = 39 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

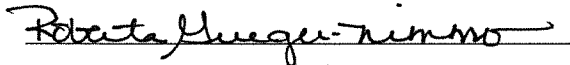
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:04
02 Std. Gas	0.080	08:05
03 Room Air	0.000	08:05
04 Std. Gas	0.079	08:05
05 Room Air	0.000	08:06
06 Std. Gas	0.080	08:06
07 Room Air	0.000	08:07

Lot No = 19817080A4
Cyl No = 24
Exp Date = 09/05/2019
County = 39 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:08
02 Std. Gas	0.080	08:08
03 Room Air	0.000	08:09
04 Std. Gas	0.079	08:09
05 Room Air	0.000	08:10
06 Std. Gas	0.079	08:10
07 Room Air	0.000	08:11

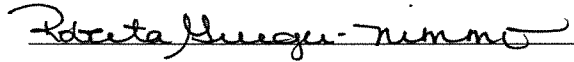
Lot No = 19817080A4

Cyl No = 24

Exp Date = 09/05/2019

County = 39

Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
06/12/2018 08:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:11
02 Std. Gas	0.080	08:12
03 Room Air	0.000	08:12
04 Std. Gas	0.080	08:13
05 Room Air	0.000	08:13
06 Std. Gas	0.080	08:14
07 Room Air	0.000	08:14

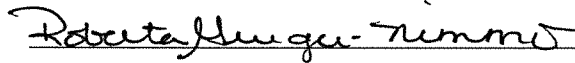
Lot No = 19817080A4

Cyl No = 24

Exp Date = 09/05/2019

County = 39

Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000