

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006678

Trail Co SO
Location: HILL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 802 psi Regulator: 800 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1CS5 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 33 Exp. Date: 5 Mar 19

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: Annual inspection.

Instrument is acceptable to be used in the field.

Roberta Hugu-Nimmis
Breath Analyst Signature

11 Jun 18
Date

Charles E. Ed
Reviewed by

12 JUN 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:08

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 33
Standard Expiration? 03/05/2019
Oper No? 888888

Flow Cal. Date: 08/22/2016
Slope 686
Intercept -611596

IR Calibration Date: 08/22/2016
 3um 9um

0th Coef(*100):	-15445	-21650
1st Coef(*100):	251854	131766
2nd Coef(*100):	3094	1378
H2O adj(mg/l*10k):	549	513

***** Printer Test End *****

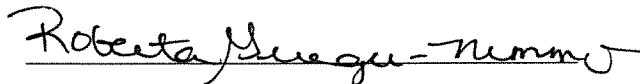

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Print Test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Sol.	0.019	14:44
03 Room Air	0.000	14:45
04 Std. Sol.	0.019	14:46
05 Room Air	0.000	14:46
06 Std. Sol.	0.019	14:47
07 Room Air	0.000	14:47

08 Sim Temp = 34.0°C

Simul Ser No = DR5143
Std Sol No = 201610A
County = 49 Oper No. = 888888

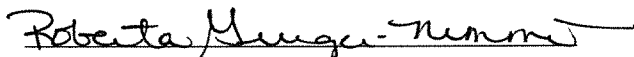
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:38
02 Std. Sol.	0.294	14:39
03 Room Air	0.000	14:39
04 Std. Sol.	0.293	14:40
05 Room Air	0.000	14:41
06 Std. Sol.	0.294	14:41
07 Room Air	0.000	14:42

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 49 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:21

Test	AC	Time
01 Room Air	0.000	14:22
02 *Subject Test	INT*	14:23
03 Room Air	0.000	14:23

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 33
Expiration Date = 03/05/2019
County = 49 Oper No. = 888888

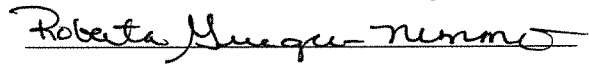

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:19

Test	AC	Time
01 Diagnostic	OK	14:20
02 Room Air	0.000	14:20
03 *Subject Test	RFI*	14:21
04 Room Air	0.000	14:21

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 33
Expiration Date = 03/05/2019
County = 49 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

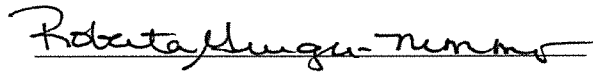

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:09
02 Std. Gas	0.080	14:10
03 Room Air	0.000	14:10
04 Std. Gas	0.080	14:11
05 Room Air	0.000	14:11
06 Std. Gas	0.080	14:11
07 Room Air	0.000	14:12

Lot No = 03417080A3
Cyl No = 33
Exp Date = 03/05/2019
County = 49 Oper No. = 888888

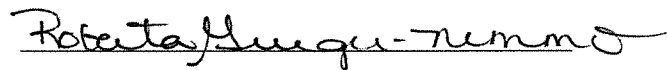

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:13
02 Std. Gas	0.080	14:13
03 Room Air	0.000	14:14
04 Std. Gas	0.080	14:14
05 Room Air	0.000	14:15
06 Std. Gas	0.081	14:15
07 Room Air	0.000	14:15

Lot No = 03417080A3
Cyl No = 33
Exp Date = 03/05/2019
County = 49 Oper No. = 888888

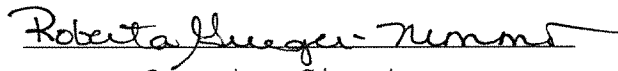
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = HILL 8164.14.00 09/16
06/11/2018 14:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:16
02 Std. Gas	0.080	14:17
03 Room Air	0.000	14:17
04 Std. Gas	0.080	14:17
05 Room Air	0.000	14:18
06 Std. Gas	0.080	14:18
07 Room Air	0.000	14:19

Lot No = 03417080A3
Cyl No = 33
Exp Date = 03/05/2019
County = 49 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC

Form 106-I8000