

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006684

West Fargo PD  
Location: WFPD

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 996 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS5 Exp. Date: 15 May 19  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 19817080A4 Cyl #: 4 Exp. Date: 5 Sep 19

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection. Completed tank sensor tare/calibration.

Instrument is acceptable to be used in the field.

Roberta Huegel-Timmo  
Breath Analyst Signature

12 Jun 18  
Date

Charles E. Ede  
Reviewed by

12 JUN. 18  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                      10:57

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            19817080A4  
Standard Cyl #?            4  
Standard Expiration?       09/05/2019  
Oper No?                    888888

Flow Cal. Date:            09/06/2016  
Slope                      709  
Intercept                  -725441

IR Calibration Date:        09/06/2016  
                                    3um                      9um

0th Coef(*100):	-16013	-23467
1st Coef(*100):	256431	135731
2nd Coef(*100):	2904	1268
H2O adj(mg/l*10k):	553	514

\*\*\*\*\* Printer Test End \*\*\*\*\*

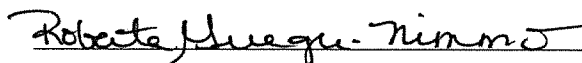

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                        11:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Sol.	0.019	11:24
03 Room Air	0.000	11:25
04 Std. Sol.	0.019	11:25
05 Room Air	0.000	11:26
06 Std. Sol.	0.019	11:27
07 Room Air	0.000	11:27

08 Sim Temp = 34.0°C

Simul Ser No = DR5143  
Std Sol No = 201610A  
County = 09                            Oper No. = 888888

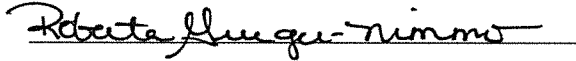

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: low AC  
          0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                        11:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Sol.	0.294	11:15
03 Room Air	0.000	11:16
04 Std. Sol.	0.294	11:17
05 Room Air	0.000	11:17
06 Std. Sol.	0.292	11:18
07 Room Air	0.000	11:18


08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 09

Oper No. = 888888

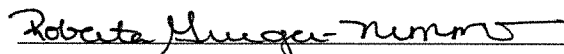

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: High AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                        11:11

Test	AC	Time
01 Room Air	0.000	11:12
02 *Subject Test	INT*	11:13
03 Room Air	0.000	11:13

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 4  
Expiration Date = 09/05/2019  
County = 09                            Oper No. = 888888

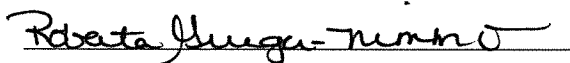

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*  
*0.10 AC + 0.05% Acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                        11:09

Test	AC	Time
01 Diagnostic	OK	11:10
02 Room Air	0.000	11:10
03 *Subject Test	RFI*	11:11
04 Room Air	0.000	11:11

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 4  
Expiration Date = 09/05/2019  
County = 09                          Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

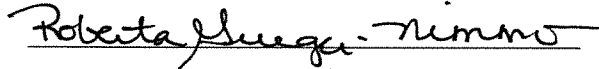

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: **RFI CHECK**

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                    8164.14.00 09/16  
06/12/2018                            10:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:58
02 Std. Gas	0.080	10:58
03 Room Air	0.000	10:59
04 Std. Gas	0.081	10:59
05 Room Air	0.000	11:00
06 Std. Gas	0.080	11:00
07 Room Air	0.000	11:00

Lot No = 19817080A4  
Cyl No = 4  
Exp Date = 09/05/2019  
County = 09                            Oper No. = 888888

*Roberta Grieger-Nimmo*

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006684  
Location = WFPD                8164.14.00 09/16  
06/12/2018                        11:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:02
02 Std. Gas	0.080	11:02
03 Room Air	0.000	11:03
04 Std. Gas	0.080	11:03
05 Room Air	0.000	11:04
06 Std. Gas	0.080	11:04
07 Room Air	0.000	11:05

Lot No = 19817080A4

Cyl No = 4

Exp Date = 09/05/2019

County = 09

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006684  
Location = WFPD      8164.14.00 09/16  
06/12/2018      11:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:06
02 Std. Gas	0.080	11:06
03 Room Air	0.000	11:07
04 Std. Gas	0.081	11:07
05 Room Air	0.000	11:07
06 Std. Gas	0.080	11:08
07 Room Air	0.000	11:08

Lot No = 19817080A4  
Cyl No = 4  
Exp Date = 09/05/2019  
County = 09      Oper No. = 888888

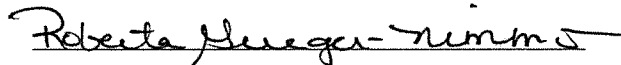

Operator Signature  
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check  
0.080 AC

Form 106-I8000