



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number <i>80-006671</i>	Instrument Location <i>Coop</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <i>Install after bringing to Crime lab for Download</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *NA*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>7/9/18</i>
---	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Robert Burgess-Humans</i>	Certified Date <i>7/9/18</i>
---	---------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = COOP 8164.14.00 09/16
07/09/2018 14:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 10
Standard Expiration? 03/05/2019
Oper No? 061123

Flow Cal. Date: 08/22/2016
Slope 703
Intercept -691868

IR Calibration Date: 08/22/2016

	3um	9um
0th Coef(*100):	-17135	-24345
1st Coef(*100):	254798	132856
2nd Coef(*100):	3386	1354
H2O adj(mg/l*10k):	610	539

***** Printer Test End *****

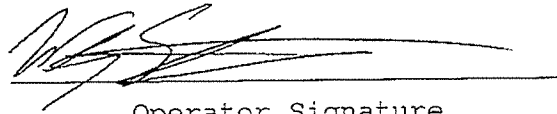

Operator Signature
WESLEY STRAIGHT

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = COOP 8164.14.00 09/16
07/09/2018 14:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:05
02 Std. Gas	0.080	14:06
03 Room Air	0.000	14:06
04 Std. Gas	0.081	14:07
05 Room Air	0.000	14:07
06 Std. Gas	0.080	14:07
07 Room Air	0.000	14:08

Lot No = 03417080A3
Cyl No = 10
Exp Date = 03/05/2019
County = 20 Oper No. = 061123

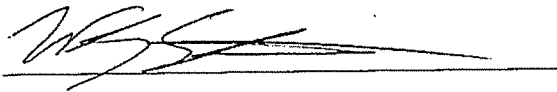

Operator Signature
WESLEY STRAIGHT

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = COOP 8164.14.00 09/16
07/09/2018 14:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:10
02 Room Air	0.000	14:10

*Invalid Test
Inhibited - RFI

Lot No = 03417080A3
Cyl No = 10
Exp Date = 03/05/2019
County = 20 Oper No. = 061123

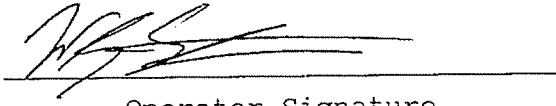
Operator Signature
WESLEY STRAIGHT

Remarks:

Form 106-I8000