



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number <i>80-006676</i>	Instrument Location <i>2051st Ave NW Mandan old jail</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>7/16/18</i>
---	------------------------

Crime Laboratory Use Only.

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Huggins-Rimmer</i>	Certified Date <i>7/17/18</i>
--	----------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = MAND 8164.14.00 09/16
07/16/2018 16:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.080	16:04
03 Room Air	0.000	16:04
04 Std. Gas	0.080	16:04
05 Room Air	0.000	16:05
06 Std. Gas	0.080	16:05
07 Room Air	0.000	16:06

Lot No = 34917080A3
Cyl No = 35
Exp Date = 02/05/2020
County = 30 Oper No. = 020611


Operator Signature
TOM SHARP

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006676
Location = MAND 8164.14.00 09/16
07/16/2018 16:07

Test	AC	Time
01 Room Air	0.000	16:10
02 Reference	RFI*	16:10
03 Room Air	0.000	16:10

*Invalid Test
Inhibited - RFI

Sub Name = RFITEST, INSTALL NA
Sub DOB = 01/01/2001
Sub Sex = Unknown Weight = NA
Test = OTE Cit = NA
Dr. Lic. = ND/NA
Lot No = 34917080A3
Cyl No = 35
Expiration Date = 02/05/2020
County = 30 Oper No. = 020611



Operator Signature
TOM SHARP

Remarks: