



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (10-2016)

Serial Number 80-006509	Instrument Location Mercer City Jail Stanton
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

NA
3/10

Field Inspector Signature 	Date 8/6/18
-------------------------------	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by 	Certified Date 8/7/18
---------------------------	---------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = STAT 8164.14.00 09/16
08/06/2018 14:15

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 007
Standard Expiration? 03/05/2019
Oper No? 120906

Flow Cal. Date: 08/20/2015
Slope 695
Intercept -685441

IR Calibration Date: 04/01/2016

	3um	9um
0th Coef(*100):	-42983	-22063
1st Coef(*100):	281397	138045
2nd Coef(*100):	2616	1525
H2O adj(mg/l*10k):	643	374

***** Printer Test End *****



Operator Signature
PAMELA PEHL

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = STAT 8164.14.00 09/16
08/06/2018 14:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Gas	0.077	14:23
03 Room Air	0.000	14:24
04 Std. Gas	0.077	14:24
05 Room Air	0.000	14:25
06 Std. Gas	0.077	14:25
07 Room Air	0.000	14:26

Lot No = 03417080A3
Cyl No = 007
Exp Date = 03/05/2019
County = 29 Oper No. = 120906

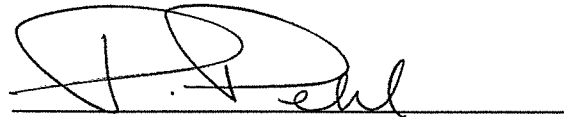

Operator Signature
PAMELA PEHL

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = STAT 8164.14.00 09/16
08/06/2018 14:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Gas	0.078	14:42
03 Room Air	RFI*	14:42
04 Room Air	RFI*	14:42

*Invalid Test
Inhibited - RFI

Lot No = 03417080A3
Cyl No = 007
Exp Date = 03/05/2019
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks: