

**INTOXILYZER® 8000 INITIAL INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-007086

**A. GENERAL Setup and Checks:**

1.  Power On  
 5 min. countdown during initial warm-up  
 1 min. countdown to begin test after standby mode
2.  Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:  
 Level 1 (Operator)  Level 2 (Field Inspector)  Level 3 (Lab Personnel)
4.  Printer Setup (Level 3,S,P)
5.  Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

1.  Internal Printer Test and Paper Feed (Level 1,P). Attach record.
2.  External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):  
 Operator ID  ND License  R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

1.  Continuous Air Blank (Level 2,A)
2.  Diagnostic (Level 2,D). Attach record.
3.  Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):  
Inst Ser#: 80-007086 DSP Ver: 1362.14  
Core Ver: 8164.15 CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

1. DVM Monitor (Level 3,D,D): 3: 12597 s: 1 c: 47.00  
9: 13403 s: 1 B: 44 0  
(RFI indicator with keyed radio): 1
2. Function (F) Keys: :  F5 – Tone  F9 - Fan  
 F6 – Green LED  F10 – Pump  
 F7 – Red LED  F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 967 CF: 1.04
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 184 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 48.0 I: 0.0  
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 660 psi P (gauge): 675 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: MP3058 Lot #: MILLI H<sub>2</sub>O Exp date: NA  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: DR7111 Lot #: 17160 Exp date: 6.8.19  
Known value: 0.020 AC  
Results: 0.018 AC 0.018 AC 0.019 AC
3. High AC (Level 1,C):  
Sim ser #: DR3375 Lot #: 17350 Exp date: 10.11.19  
Known value: 0.300 AC  
Results: 0.298 AC 0.299 AC 0.300 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR7089 Lot #: ICS#5 Exp date: 5-15-19  
Known value: 0.05% ACETONE + 0.10 AC  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR3846 Lot #: ACE 1 Exp date: NA  
Known value: 0.05% ACETONE  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR3455 Lot #: MET 2 Exp date: NA  
Known value: 0.10% METHANOL  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR5189 Lot #: IPA 2 Exp date: NA  
Known value: 0.10% ISOPROPANOL  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):  
Ave: 0.0811 Std Dev: 0.0004 Rel Std Dev: 0.56%

Maintenance/Repairs/Notes: INITIAL INSPECTION ON NEW  
INSTRUMENT RECEIVED FROM CMI.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

Instrument is acceptable to be used in the field.

Charles E. Edin  
Field Inspector's Signature

9/6/2018  
Date

Roberta Mungie-Tumms  
Reviewed by

9/6/2018  
Date

Intoxilyzer Test Record and Checklist  
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-007086  
Location = TOXL 8164.15.00 08/18  
09/05/2018 14:20

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMN

24 characters per line  
abcdefghijklmnopqrstuvwx  
ABCDEFGHIJKLMN

32 characters per line  
abcdefghijklmnopqrstuvwxy123456  
ABCDEFGHIJKLMN

42 characters per line  
abcdefghijklmnopqrstuvwxy1234567890abcdef  
ABCDEFGHIJKLMN

Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On MenFull? No  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 19817080R4  
Standard Cyl #? 21  
Standard Expiration? 09/05/2019  
Oper No? 666666

Flow Cal. Date: 06/25/2018  
Slope: 671  
Intercept: -609390

IR Calibration Date: 06/25/2018

	3um	9um
0th Coef(*100):	-37724	-21405
1st Coef(*100):	229685	127813
2nd Coef(*100):	4080	1789
H2O adj(mg/l*10k):	1433	679

\*\*\*\*\* Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

Internal Print  
Test  
Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-007086  
Location = TOXL                8164.15.00 08/18  
09/05/2018                       14:21

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?         No  
Display Volume?             No  
Disable On Memfull?         No  
# of Print Copies?          1  
Select Std (D/W/I)?         Dry  
Standard Value?             0.080  
Standard Lot #?             19817080A4  
Standard Cyl #?             21  
Standard Expiration?        09/05/2019  
Oper No?                    666666

Flow Cal. Date:             06/25/2018  
Slope                       671  
Intercept                  -609390

IR Calibration Date:        06/25/2018  
                              3um                    9um

-----  
0th Coef(\*100):            -37724            -21405  
1st Coef(\*100):            229685            127813  
2nd Coef(\*100):            4080               1789  
H2O adj(mg/l\*10k):        1433               679

\*\*\*\*\* Printer Test End \*\*\*\*\*

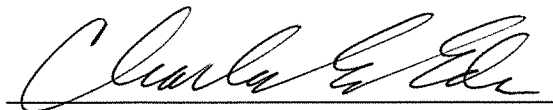

Operator Signature  
CHARLES EDER

Remarks: *EXTERNAL PRINT*

*TEST*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      14:25

Test	AC	Time
01 Room Air	0.000	14:26
02 *Subject Test	REF*	14:26
03 Room Air	0.000	14:26

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = R KEY REFUSAL  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08      Oper No. = 666666

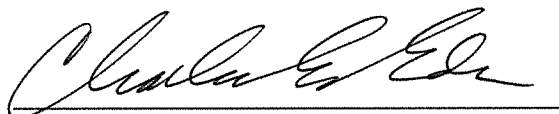

Operator Signature  
CHARLES EDER

Remarks: *BARCODE READER*

*R KEY REFUSAL*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      14:45

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

DIAGNOSTIC  
CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                    8164.15.00 08/18  
09/05/2018                            15:00

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:00
02 Std. Sol.	0.000	15:01
03 Room Air	0.000	15:02
04 Std. Sol.	0.000	15:02
05 Room Air	0.000	15:03
06 Std. Sol.	0.000	15:03
07 Room Air	0.000	15:04

08 Sim Temp = 34.0°C

Simul Ser No = MP3058  
Std Sol No = MILLIQ H2O  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

ZERO AC  
0.000 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      15:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:08
02 Std. Sol.	0.018	15:09
03 Room Air	0.000	15:09
04 Std. Sol.	0.018	15:10
05 Room Air	0.000	15:11
06 Std. Sol.	0.019	15:11
07 Room Air	0.000	15:12

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      15:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:14
02 Std. Sol.	0.298	15:15
03 Room Air	0.000	15:16
04 Std. Sol.	0.299	15:16
05 Room Air	0.000	15:17
06 Std. Sol.	0.300	15:18
07 Room Air	0.000	15:18

08 Sim Temp = 34.0°C

Simul Ser No = DR3375  
Std Sol No = 17350  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *HIGH AC*  
                    *0.300 AC*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                 8164.15.00 08/18  
09/05/2018                        15:21

Test	AC	Time
01 Room Air	0.000	15:22
02 *Subject Test	INT*	15:23
03 Room Air	0.000	15:24

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                 Weight = 150  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.05% ACETONE + 0.10 AR*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                 8164.15.00 08/18  
09/05/2018                        15:24

Test	AC	Time
01 Room Air	0.000	15:25
02 *Subject Test	INT*	15:25
03 Room Air	0.000	15:26

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                 Weight = 150  
Test = DUI                         Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                         Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: 0.05% ACETONE  
Interferent check

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      15:34

Test	AC	Time
01 Room Air	0.000	15:35
02 *Subject Test	INT*	15:35
03 Room Air	0.000	15:36

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08      Oper No. = 666666

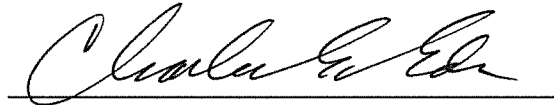

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*

*0.10% METHANOL*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                8164.15.00 08/18  
09/05/2018                        15:37

Test	AC	Time
01 Room Air	0.000	15:38
02 *Subject Test	INT*	15:38
03 Room Air	0.000	15:39

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                        Oper No. = 666666

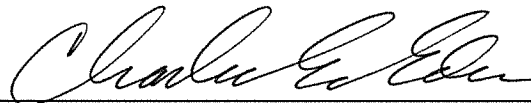

Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10% ISOPROPANOL*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer          Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL                8164.15.00 08/18  
09/05/2018                      15:43

Test	AC	Time
01 Room Air	AMB*	15:44
02 Room Air	0.000	15:45

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                Weight = 150  
Test = DUI                        Cit = AMBIENT FAIL  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*AMBIENT FAIL*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007086  
Location = TOXL            8164.15.00 08/18  
09/05/2018                            15:46

Test	AC	Time
01 Room Air	0.000	15:47
02 Invalid Sample	X.XXX	15:47
03 Room Air	0.000	15:48

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                    Cit = INVALID SAMPLE  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                    Oper No. = 666666

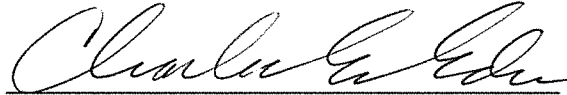

Operator Signature  
CHARLES EDER

Remarks: *INVALID SAMPLE*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-007086  
Location = TOXL                              8164.15.00 08/18  
09/05/2018    15:48

Test	AC	Time
01 Room Air	0.000	15:49
02 *Subject Test	ABT*	15:49
03 Room Air	0.000	15:50

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                              Weight = 150  
Test = DUI                              Cit = SEQUENCE ABORT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                              Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

SEQUENCE  
ABORTED

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-007086  
Location = TOXL                      8164.15.00 08/18  
09/05/2018    15:50

Test	AC	Time
01 Room Air	0.000	15:51
02 Subject Test 1	0.038	15:51
03 Room Air	0.000	15:52
04 Reported AC	0.038	15:51

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                      Cit = ABA MODE  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                      Oper No. = 666666

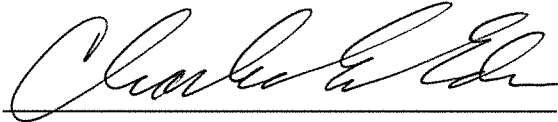

Operator Signature  
CHARLES EDER

Remarks:

*ABA TEST*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007086  
Location = TOXL                    8164.15.00 08/18  
09/05/2018                            15:52

Test	AC	Time
01 Diagnostic	OK	15:53
02 Room Air	0.000	15:54
03 Subject Test 1	0.012	15:54
04 Room Air	0.000	15:57
05 Std. Gas	0.090*	15:58
06 Room Air	0.000	15:58

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = CAL CHK OUT TOL  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
          OUT OF TOLERANCE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000           SN 80-007086  
Location = TOXL                   8164.15.00 08/18  
09/05/2018                               15:59

Test	AC	Time
01 Diagnostic	OK	16:00
02 Room Air	0.000	16:01
03 *Subject Test	RFI*	16:01
04 Room Air	0.000	16:02

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                   Weight = 150  
Test = DUI                           Cit = RFI DETECT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                           Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

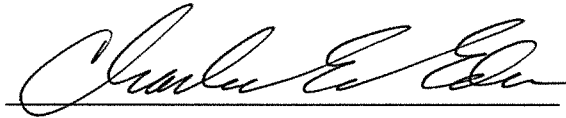

Operator Signature  
CHARLES EDER

Remarks:

*RFI DETECT*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-007086  
Location = TOXL                                      8164.15.00 08/18  
09/05/2018    16:04

Test	AC	Time
01 Diagnostic	OK	16:05
02 Room Air	0.000	16:05
03 *Subject Test	0.000*	16:08
04 Room Air	0.000	16:10
05 Std. Gas	0.083	16:12
06 Room Air	0.000	16:13
07 *Subject Test	0.000*	16:16
08 Room Air	0.000	16:19

No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                                      Weight = 150  
Test = DUI                                      Cit = DEFICIENT SAMP  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                                      Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: DEFICIENT SAMPLES

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-007086  
Location = TOXL                              8164.15.00 08/18  
09/05/2018    16:23

Test	AC	Time
01 Diagnostic	OK	16:24
02 Room Air	0.000	16:25
03 Reference	IPS*	16:25
04 Room Air	0.000	16:25

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                              Weight = 150  
Test = DUI                              Cit = IMPROPER SAMPLE  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                              Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*IMPROPER  
SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
 NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
 North Dakota Model 8000                      SN 80-007086  
 Location = TOXL                                  8164.15.00 08/18  
 09/05/2018    16:26

Test	AC	Time
01 Diagnostic	OK	16:27
02 Room Air	0.000	16:27
03 Subject Test 1	0.252*	16:27
04 Room Air	0.000	16:30
05 Std. Gas	0.082	16:31
06 Room Air	0.000	16:32
07 Subject Test 2	0.297*	16:33
08 Room Air	0.000	16:34

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
 Sub DOB = 01/01/1982  
 Sub Sex = Female                                  Weight = 150  
 Test = DUI                                  Cit = DIFF TOO GREAT  
 Dr. Lic. = ND/DIS821456  
 Lot No = 19817080A4  
 Cyl No = 21  
 Expiration Date = 09/05/2019  
 County = 08                                  Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
 CHARLES EDER

Remarks: *DIFFERENCE*  
*TOO GREAT*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                    8164.15.00 08/18  
09/05/2018                            16:35

Test	AC	Time
01 Diagnostic	OK	16:36
02 Room Air	0.000	16:36
03 Subject Test 1	0.250	16:36
04 Room Air	0.000	16:39
05 Std. Gas	0.080	16:40
06 Room Air	0.000	16:41
07 Subject Test 2	0.248	16:42
08 Room Air	0.000	16:43
09 Reported AC	0.248	16:42

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = CMS TEST  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 21  
Expiration Date = 09/05/2019  
County = 08                            Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

CMS TEST



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.15.00 08/18  
09/05/2018      16:46

Test	AC	Time
01 Room Air	0.000	16:46
02 Std. Gas	0.081	16:47
03 Room Air	0.000	16:47
04 Std. Gas	0.081	16:48
05 Room Air	0.000	16:48
06 Std. Gas	0.080	16:49
07 Room Air	0.000	16:49
08 Std. Gas	0.081	16:49
09 Room Air	0.000	16:50
10 Std. Gas	0.082	16:50
11 Room Air	0.000	16:51
12 Std. Gas	0.082	16:51
13 Room Air	0.000	16:52
14 Std. Gas	0.081	16:52
15 Room Air	0.000	16:53
16 Std. Gas	0.081	16:53
17 Room Air	0.000	16:54
18 Std. Gas	0.082	16:54
19 Room Air	0.000	16:55
20 Std. Gas	0.081	16:55
21 Room Air	0.000	16:56
22 Std. Gas	0.081	16:56
23 Room Air	0.000	16:57
24 Std. Gas	0.081	16:57
25 Room Air	0.000	16:58
26 Std. Gas	0.081	16:58
27 Room Air	0.000	16:58
28 Std. Gas	0.081	16:59
29 Room Air	0.000	16:59
30 Std. Gas	0.081	17:00
31 Room Air	0.000	17:00
32 Std. Gas	0.081	17:01
33 Room Air	0.000	17:01
34 Std. Gas	0.081	17:02
35 Room Air	0.000	17:02
36 Std. Gas	0.081	17:03
37 Room Air	0.000	17:03
38 Std. Gas	0.081	17:04
39 Room Air	0.000	17:04
40 Std. Gas	0.081	17:04
41 Room Air	0.000	17:05
42 Cal Check Stats:		
43 Average	0.0811	
44 Std Dev	0.0004	
45 Rel Std Dev	0.56%	



Operator Signature  
CHARLES EDER

Remarks:

STABILITY TEST  
0.080 AC

Form 106-I8000

Lot: 19817080A24  
CEE

CYL: 021

Exp: 9-5-2019