



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number <i>80-006686</i>	Instrument Location <i>Walsh County</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <i>Inter Check</i>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *none*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Brad [Signature] #1442</i>	Date <i>Sept-17-2018</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Huque-Nimmo</i>	Date <i>Sept-17-2018</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Roberta Huque-Nimmo* Certified Date: *Sept-17-2018*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = GRAF 8164.14.00 09/16
09/17/2018 02:49

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 22
Standard Expiration? 09/05/2019
Oper No? 130890

Flow Cal. Date: 09/07/2016
Slope 770
Intercept -749410

IR Calibration Date: 09/06/2016
 3um 9um

0th Coef(*100): -14519 -24061
1st Coef(*100): 252211 131592
2nd Coef(*100): 3527 1490
H2O adj(mg/l*10k): 522 594

***** Printer Test End *****

 # 1442

Operator Signature
BRADLEY WINGERTER

Remarks: Intox check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = GRAF 8164.14.00 09/16
09/17/2018 03:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	03:10
02 Std. Gas	0.081	03:10
03 Room Air	0.000	03:11
04 Std. Gas	0.080	03:11
05 Room Air	0.000	03:12
06 Std. Gas	0.080	03:12
07 Room Air	0.000	03:13

Lot No = 19817080A4

Cyl No = 22

Exp Date = 09/05/2019

County = 50

Oper No. = 130890

Bradley Wingarter #1442

Operator Signature
BRADLEY WINGERTER

Remarks: Intox check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006686
Location = GRAF 8164.14.00 09/16
09/17/2018 03:14

Test	AC	Time
01 Room Air	0.000	03:17
02 Reference	RFI*	03:17
03 Room Air	0.000	03:17

*Invalid Test
Inhibited - RFI

Sub Name = DOE, JOHN DAVE
Sub DOB = 09/29/1999
Sub Sex = Male Weight = 199
Test = DUI Cit = 1234567
Dr. Lic. = ND/DOE991234
Lot No = 19817080A4
Cyl No = 22
Expiration Date = 09/05/2019
County = 50 Oper No. = 130890

Bradley Wingarter # 1442

Operator Signature
BRADLEY WINGERTER

Remarks: *Intox Check*