



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006673	Instrument Location Burke County Community Room
Reason for Install/Repair (Temp. Sheriff's Office)	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

*Completed
See
attached
tests
9/20
2018
NA
2018*

Field Inspector Signature 	Date 9/27/18
-------------------------------	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 10/2/2018
---------------------------	------------------------------------

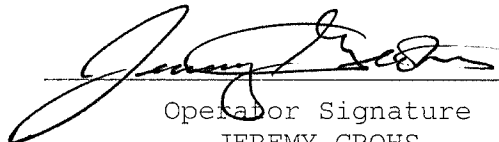
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
09/27/2018 11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:22
02 Std. Gas	0.081	11:22
03 Room Air	0.000	11:23
04 Std. Gas	0.081	11:23
05 Room Air	0.000	11:23
06 Std. Gas	0.081	11:24
07 Room Air	0.000	11:24

Lot No = 03417080A3
Cyl No = 9
Exp Date = 03/05/2019
County = 07 Oper No. = 130230


Operator Signature
JEREMY GROHS

Remarks: *Good Test*

I certify that the attached is a true and correct copy of the Intoxilyzer test record.

Print Name and Badge # *Jeremy Grohs #4170*

Signature 

Date *9/27/18*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

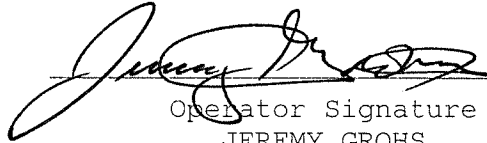
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BURK 8164.14.00 09/16
09/27/2018 11:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:29
02 Room Air	0.000	11:29

*Invalid Test
Inhibited - RFI

Lot No = 03417080A3
Cyl No = 9
Exp Date = 03/05/2019
County = 07 Oper No. = 130230

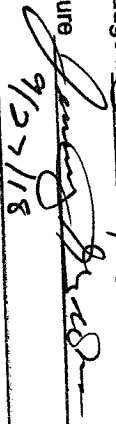

Operator Signature
JEREMY GROHS

Remarks:

Good Test

Form 106-I8000

I certify that the attached is a true and correct copy of the Intoxilyzer test record.

Print Name and Badge # Jeremy Grohs #4170
Signature 
Date 9/27/18