



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number <u>80-006690</u>	Instrument Location <u>MNAF</u>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <u>20 Oct 18</u>
-------------------------------	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date <u>10-11-2018</u>
---------------------------	-------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
10/02/2018 14:33

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 8
Standard Expiration? 09/05/2019
Oper No? 132026

Flow Cal. Date: 08/23/2018
Slope 677
Intercept -593298

IR Calibration Date: 08/23/2018

	3um	9um
0th Coef(*100):	-8380	-21663
1st Coef(*100):	265142	130429
2nd Coef(*100):	3933	1429
H2O adj(mg/l*10k):	412	539

***** Printer Test End *****

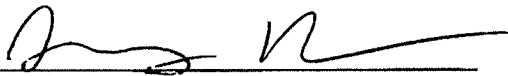

Operator Signature
JEREMY HOMAN

Remarks: *Install*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
10/02/2018 16:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.079	16:04
03 Room Air	0.000	16:04
04 Std. Gas	0.079	16:05
05 Room Air	0.000	16:05
06 Std. Gas	0.079	16:06
07 Room Air	0.000	16:06

Lot No = 19817080A4
Cyl No = 8
Exp Date = 09/05/2019
County = 64 Oper No. = 132026

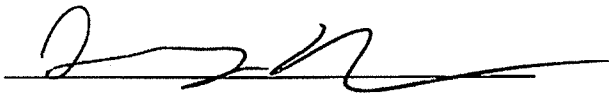

Operator Signature
JEREMY HOMAN

Remarks: *Install*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
10/02/2018 16:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	16:08
02 Room Air	0.000	16:08

*Invalid Test
Inhibited - RFI

Lot No = 19817080A4
Cyl No = 8
Exp Date = 09/05/2019
County = 64 Oper No. = 132026



Operator Signature
JEREMY HOMAN

Remarks: *Install*