

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Contribution Co. of the Co.	
Serial Number 00 70 86 Instrument Location CISDON	
Reason for Install/Repair	
Install After Receiving From Crime Laboratory Install After Location Change	
Other (Specify)	
Check When Done:	
1. Surge Protector Installed/Property Grounded.	
2. Telephone Line Connected to Intoxilyzer® 8000.	
プ,3. Breath Tube Heated.	
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).	
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).	
6. Run Tests:	
🗹 A. Print Test (Level1, Function P).	
B. ACA Test (Level 1, Function C).	·
C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio Du	ring Test).
図 7. Repair and/or Maintenance Performed (if any): 人つガー	
8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the In	ntoxilyzer® for Use.
9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Age	ncy.
10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair C 104-G), Print Test, ACA Test, and RFI Test.	heckout (SFN59281, Form
Field Inspector Signature 1	Date -
The Japan	Date 12-20-18
Crime Laboratory Use Only	
This installation has been reviewed and the instrument is approved to be used for the analysis of breath to deter the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crin County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.	mine alcohol concentration from ne Laboratory Division, in the
Reviewed/Certified By	Certified Date
Foberta Sueger ninns	12-27-18
Note: See note from field inspector Tim Jordan pg 40f 5 3000,2	d Form 120-6
Note: See note from field inspector Tim Jordan pg 40f 5 and 100 fly 10-27-18 Relevant	

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007086
Location = LISB 8164.16.00 09/18
12/20/2018 19:57

********* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()__+?

Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer (Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 17618080A3 Standard Cyl #? 66 Standard Expiration? 08/05/2020 Oper No? 020607

Flow Cal. Date: 10/16/2018 Slope 688 Intercept -653049

IR Calibration Date: 10/16/2018

3um 9um

Oth Coef(*100): -47466 -23263
1st Coef(*100): 233222 130299
2nd Coef(*100): 2622

1St Coet(*100): 233222 13029 2nd Coef(*100): 3639 1618 H2O adj(mg/l*10k): 1280 608

******* Printer Test End *******

Operator Signature
TIMOTHY JORDAN

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = LISB
12/20/2018

Alcohol Analyzer SN 80-007086 8164.16.00 09/18 19:58

DRY CAL CHECK

Test			AC	Time
01	Room	Air	0.000	19:59
02	Std.	Gas	0.081	19:59
03	Room	Air	0.000	20:00
04	Std.	Gas	0.081	20:00
05	Room	Air	0.000	20:01
06	Std.	Gas	0.081	20:01
07	Room	Air	0.000	20:02

Lot No = 17618080A3

Cyl No = 66

Exp Date = 08/05/2020

County = 37

Oper No. = 020607

Operator Signature TIMOTHY JORDAN

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = LISB
12/20/2018

Alcohol Analyzer SN 80-007086 8164.16.00 09/18 20:03

DRY CAL CHECK

Test	AC	Time
01 Room Air 02 Room Air	RFI* 0.000	20:04

*Invalid Test Inhibited - RFI

Lot No = 17618080A3

Cyl No = 66

Exp Date = 08/05/2020

County = 37

Oper No. = 020607


Operator Signature TIMOTHY JORDAN

Remarks:

Form 106-I8000

There was an issue with our Phone line that caused the difference in Jates on the form 120-G

Tim Jordan Badge # 1521

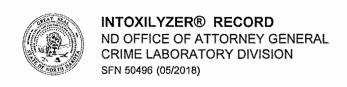

Intoxilyzer® Serial Num	nber 07086		Location LIShon	P.D
Gas Lot Number,		Gas Cylinder Number		Gas Expiration Date
Test Date	Chemical Test Operator Number			or's Name Name, First, MI)

170100	0 / 1	101703/240
Test Date	Chemical Test Operator Number	Operator's Name (PRINT Last Name, First, MI)
12-10-18	020607	Jordan Timothy A Machine Install
12-20-18	020607	Jordan Timothy A Print test
12-20-18	02-06-07	Jordan Timothy A Machine Install Jordan Timothy A Print test Jordan Timothy A RFI test
		J
ANALYSI AND SAN TO SEE SEE SEE SEE SEE SEE SEE SEE SEE SE		

		·
		·
		·
- 1200 -		